


Cement Mixing & Transfer Pack

for Use with CONCORD-Plus™ Bone Cement

1

Remove the threaded luer cap from the syringe

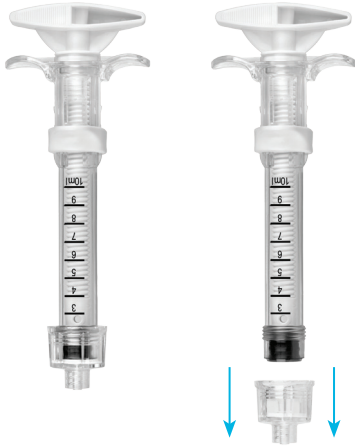

2

Pour the powder, then liquid into the cement mixer and **mix for 40 Seconds**



3

Remove the lid and insert the threaded plunger

Rotate the threaded plunger clockwise until it reaches the cement




4

Lower the base of the mixer and continue advancing the threaded plunger to remove all remaining air



5



Insert the syringe into the mixer and load with cement by continuing to advance the threaded plunger

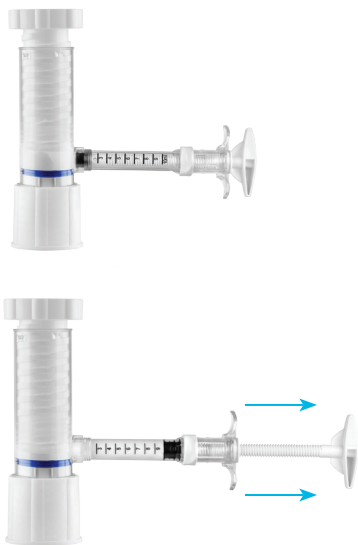

Note: Do not fill beyond the 10ml marking on the syringe

6

Remove the syringe and engage the threads by pulling the white tabs proximally towards the clear tabs and reattach the threaded luer cap to the syringe

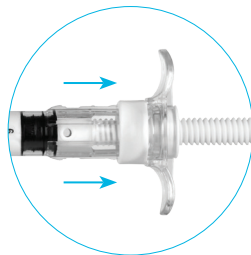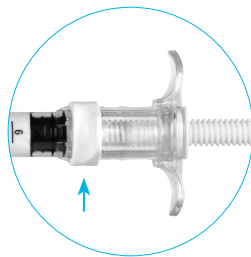

7

Cement Transfer

- Connect a filler delivery needle to the syringe and rotate the handle clockwise to fill the needle with cement
- Expel 1 cm of cement from the end of the filler needle ("dry plug")
- Continue filling remaining needles
- Use the enclosed plungers for cement injection

