



## INSTRUCTIONS FOR USE STERISHIELD™ SINGLE LAYER PROTECTIVE AMNION PATCH



Stores at room temperature. No refrigeration necessary.

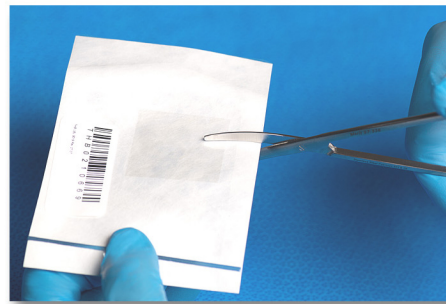

Trim through the sterile inner package if needed.

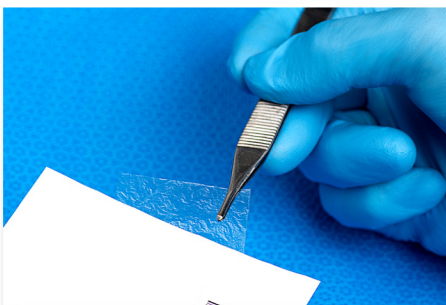


Take graft directly from package to surgical site.



Place using orientation notch as guide: in the upper right hand corner. Ensures epithelial layer faces out and stromal (sticky) side faces wound.



Apply dry, ideally with forceps.

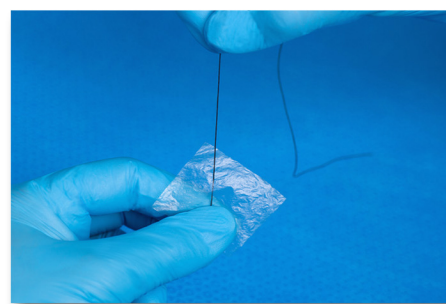

Sutures or adhesives may be used if indicated.

### Other Comments:

Wait to open inner package until ready to place graft.  
Rehydrates quickly and becomes more difficult to handle when wet.  
Do not “play with” product too much – better to place it and leave it.