

CLINICAL STUDY SUMMARY

Minimally invasive sacroiliac joint fusion using a novel hydroxyapatite-coated screw system improves functional outcomes in patients with sacroiliitis at two year follow-up

Alex Mohit, MD, PhD and Torrey Shirk

Interdisc Neurosurg 22(December 2020):100834, 2020.

OBJECTIVE: The aim of the study was to understand the clinical outcomes of the SI-LOK[®] hydroxyapatite-coated (HA-coated) titanium screw for surgical treatment of SI joint dysfunction.


METHOD: A retrospective Institutional Review Board-exempt chart review of 45 consecutive patients who underwent minimally invasive SI joint fusion with the SI-LOK[®] Sacroiliac Joint Fusion System was performed. Patients were diagnosed based on North American Spine Society guidelines and evidence-based criteria. Clinical assessments were collected, evaluated, and compared preoperatively and 3, 6, 12, and 24 months postoperatively.




SI-LOK[®] Sacroiliac Joint Screws



SI-LOK[®] 10×60mm slotted
HA-coated SI Joint Screw

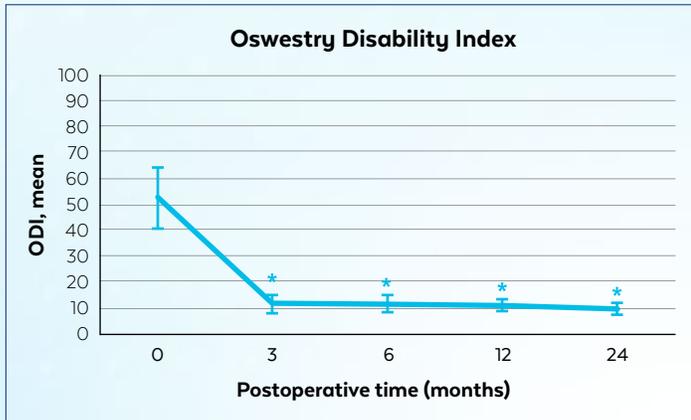

Results show a decrease in ODI scores from baseline that is sustained at 3, 6, 12, and 24 months.
*P < 0.001 compared to baseline.

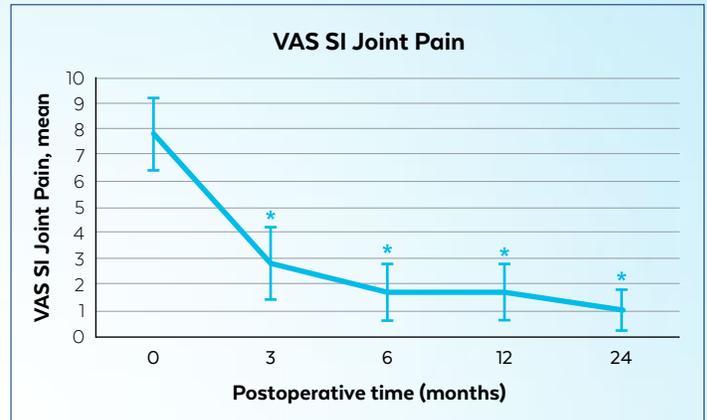

Results show a decrease in VAS SI joint pain from baseline at 3, 6, 12, and 24 months.
*P < 0.001 compared to baseline.

RESULTS:

- Of the 44 patients, nine underwent bilateral SI joint fusion; the remaining 35 fusions were unilateral. Screw size ranged from 10×35 to 10×50mm.
- Mean preoperative visual analog scale (VAS) SI pain scores decreased significantly by a mean of 6.1 points 12 months postoperatively and 6.8 points 24 months postoperatively (P < 0.001).
- Mean Oswestry Disability Index (ODI) scores significantly improved from 52.3% at baseline to 11.3% at 3 months, 11.5% at 6 months, 10.9% at 12 months, and 9.5% at 24 months, leading to improvements of 41.0, 40.8, 41.4, and 42.8 points, respectively.

CONCLUSION:

In this study, clinical outcomes of SI joint fusion using the SI-LOK® Sacroiliac Joint Fusion System to treat sacroiliitis demonstrated significant decreases in VAS SI and ODI scores at 3, 6, 12, and 24 months postoperatively.



Scan the QR code to download the article or visit globusmedical.com/si-lok-mohit-study

For more information on SI-LOK®, please visit GlobusMedical.com/si-lok

GMSS81
11.20 Rev A

