

PLEXIS[®] BONE VOID FILLING SYSTEM

1



- Open sterile package
- Move cap to recess in center of tray
- Place mixing unit in center of cap

2



- Place funnel onto mixing chamber and add powder
- Pour liquid monomer evenly into mixing chamber and begin timing

3




- Attach cap to mixing chamber and tighten lower half firmly. Crank handle should be in the fully extended position

4



- At 20 seconds, mix for 60 seconds, alternating 3 clockwise and 3 counter-clockwise revolutions. Mixing speed should be one turn per second

5

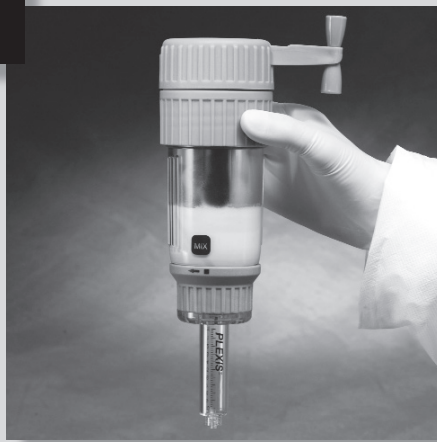

- Allow material to sit for 60 seconds after mixing

6



- Attach angled delivery tube
- Rotate collar clockwise until it locks in the "LOAD SYRINGE" position

7



- Turn crank handle clockwise until syringe is completely filled with material

8



- Snap crank handle into closed position to activate the delivery plunger

9



- Rotate crank handle clockwise to dispense. Each turn will deliver approximately 1/3 cc of material
- To stop flow of material, rotate crank handle counter-clockwise 1 full turn

