

514099-508
(REV. A)

HARVEST® ADIPREP® ACCESSORY KIT

3/2023



HARVEST® ADIPREP® ACCESSORY KIT
51496
INSTRUCTIONS FOR USE
GLOBUS MEDICAL 514099-508



GLOBUS
MEDICAL

GLOBUS MEDICAL, INC.

Valley Forge Business Center
2560 General Armistead Avenue
Audubon, PA 19403
USA

Customer Service:

Phone 1-866-GLOBUS1 (OR)

1-866-456-2871

Fax 1-866-GLOBUS3 (OR)

1-866-456-2873

WITHIN THE UNITED STATES ONLY

ENGLISH

HARVEST® ADIPREP® ACCESSORY KIT INSTRUCTIONS FOR USE

DESCRIPTION

This accessory kit is to be used in conjunction with the SmartPrep® system and the AdiPrep® Adipose Concentration System. For additional details on either the SmartPrep® system or the AdiPrep® system, see the SmartPrep® system operator's manual or the instructions for use that comes with the AdiPrep® procedure pack.

ACCESSORY KIT COMPONENTS

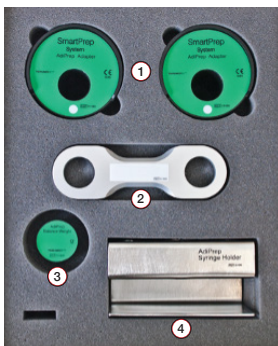


Figure 1: AdiPrep® Accessory Kit

Component Name	Description	Quantity
1 AdiPrep® adapters	The adapters that must be used when processing adipose tissue in the SmartPrep® system.	2
2 2-place process disposable holder	For use with the workstation cart to hold AdiPrep® process disposables or AdiPrep® balance weights. The back of the holder has adhesive that is used for installation on the workstation cart.	1
3 AdiPrep® balance weight	The balance weights for use when processing adipose tissue in the SmartPrep® system.	1
4 Syringe stand	The syringe stand holds syringes and can be autoclaved.	1

PROCESSING THE ADIPOSE TISSUE IN THE SMARTPREP® SYSTEM

The AdiPrep® Accessory Kit that contains the two AdiPrep® adapters is required to operate the SmartPrep® system with the AdiPrep® process disposable.

1. Install the two adapters into the rotor buckets with the white dots facing the center of the rotor, as shown in Figure 2.



Figure 2: White dots facing the center of the rotor

2. Ensure that each adapter is properly seated, as shown in Figure 3.

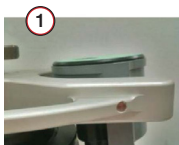


Figure 3: Adapter seated correctly (1) and adapter seated incorrectly (2)

3. Complete one of the following steps to load the system properly and avoid an imbalance:

- Load an AdiPrep® process disposable into one of the adapters, and load the AdiPrep® balance weight into the other adapter, as shown in Figure 4.



Figure 4: Loading one AdiPrep® process disposable

- Load AdiPrep® process disposables of equal volume into each adapter, as shown in Figure 5.

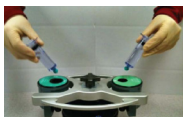


Figure 5: Loading two AdiPrep® process disposables

4. Close the lid and verify that the LID OPEN indicator is not illuminated.

5. Complete the following steps if you are using the SmartPrep® 2 System:

- Press the PRP button. The window below the TIMING indicator displays the remaining time of the processing cycle.
- When the display indicates 10 minutes remaining (approximately 4 minutes have passed), press the STOP button.
- When the rotor has stopped, press the LID button to open the lid, and remove the process disposables.

6. Complete the following steps if you are using the SmartPrep® 3 System:

- Press the AdiPrep® button.
- When the processing cycle is complete and the rotor has stopped, press the LID button to open the lid, and remove the process disposables.

INSTALLING THE 2-PLACE PROCESS DISPOSABLE HOLDER

Complete the following steps to install the 2-place process disposable holder on the workstation cart (not included):

- Remove the protective backing from the adhesive on the back of the 2-place process disposable holder. Figure 6 shows the front and back of the holder.



Figure 6: Front and back of the 2-place process disposable holder

- Position the holder on the workstation cart with adhesive facing down, as shown in Figure 7, and press it firmly into place.



Figure 7: 2-Place process disposable holder installed on the workstation cart

LATEX STATEMENT

This product is not made with natural rubber latex.

RETURN OF USED PRODUCT

If for any reason this product must be returned to Globus Medical, a returned authorization (an RA number) is required from Globus Medical prior to shipping.

IT IS THE RESPONSIBILITY OF THE HEALTH CARE INSTITUTION TO ADEQUATELY PREPARE AND IDENTIFY THE PRODUCT FOR RETURN SHIPMENT.

Please contact your local representative for information regarding returned goods and product complaints.

SYMBOL TRANSLATION			
	CATALOGUE NUMBER		COMPONENTS STERILIZED USING MULTIPLE METHODS
	LOT NUMBER		CONSULT INSTRUCTIONS FOR USE
	CAUTION		MANUFACTURER
	SINGLE USE ONLY		USE BY (YYYY-MM-DD)
	DO NOT RESTERILIZE		TEMPERATURE RANGE
	PRODUCT QUANTITY		DO NOT USE IF PACKAGE IS DAMAGED
	PRESCRIPTION ONLY		