# OPERATION MANUAL

**Rx Only** MADE IN JAPAN

OM-SH0911EN 000

Surgical Attachment

# P300 Attachment

Standard Attachment 300

Slim Attachment 300

These attachments are to be used exclusively with our total surgical system Primado2. We recommend that prior to use, you carefully read this document regarding "Precaution for Handling and Operation", "Connection method", "Operation method", "Check before Operation", "Periodical Maintenance Check" and "Reprocessing" so that you can continue using the attachments in the future.

In addition, keep this operation manual in a place where a user can see to it at any given time.

\* In this operation manual, bur and drill are collectively called Cutting Accessory(ies).

\* The reference number of the Operation Manual of the related products is as follows:

Operation Manual Title	Operation Manual No.
Total Surgical System Primado2	OM-SE0028E
Total Surgical System Primado2 Motor	OM-SE0029E
NSK Sterile Cutting Accessories for P300 Attachment	OM-SZ0959MA
CUTTING ACCESSORIES - P300 ATTACHMENT CORRESPONDENCE TABLE	OM-SZ0958MA
Cleaning Tool for P300 Attachment	OM-SZ0907EN

**\* See the "14 Symbols" section on page 68 for reference.**

# 1 User and Indications for Use

- User : Qualified Professionals (Surgeon with knowledge and skills required for operating this product).
- Indications for Use : The P300 Attachment is intended to be used with electric and pneumatic surgical instruments and cutting accessories for cutting, drilling, removal, and shaping of bone in the procedures of: Neuro, Spine, Orthopedic, ENT, Maxillofacial, and General Surgery.
- Intended patient population : The user shall determine the patient.

## 2 Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 <b>DANGER</b>	Hazard that could result in personal death or serious injury if the safety instructions are not correctly followed.
 <b>WARNING</b>	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
 <b>CAUTION</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
<b>NOTICE</b>	General product specification information highlighted to avoid product malfunction and performance reduction.

## DANGER

- DO NOT use this product in a room where there is a danger of explosion or in the vicinity of flammable substances. It is especially important not to use this product on a patient to whom a flammable anesthetic agent (dinitrogen monoxide) has been administered.
- Never disassemble or modify the product. Disassembly or modification may cause personal injury or damage to the motor (Including motor cord) or attachment.

## WARNING

- This is a medical system; do not allow anybody to use it other than surgeons who are familiar with the procedures. Use this system in accordance with its indications for use and proper method of use.
- When operating the product, always consider the safety of the patient.
- The Cutting Accessory is for single use only. DO NOT reuse or reprocess it. If you reuse or reprocess it, the following risks may occur:
  - Infection.
  - Personal injury due to damaged Cutting Accessory.
  - System defect due to damaged/deformed Cutting Accessory.
- Irrigate appropriately while cutting in order to prevent necrosis of bones and tissues.
- The attachment should be cleaned, lubricated and sterilized after use.
- Clean the attachment immediately after use. If blood remains on the product, product failure or overheating may occur due to blood coagulation and product corrosion. This may cause thermal injuries. Also, if the attachment is autoclaved with blood remaining, sterilization failure may occur, resulting in infection to the patient and/or user. (See to "12 Reprocessing").

**WARNING**

- Be sure to check for rust on the appearance of the attachment, Irrigation Nozzle and hood/beak before use. If there is any rust, stop using them, contact your Authorized NSK dealer. Then, dispose of them according to "17 Disposing Product". About the Cutting Accessories, see the operation manual of NSK Sterile Cutting Accessories for P300 Attachment. Using an attachment, Irrigation Nozzle or hood/beak with rust may lead to adverse sequelae due to potential toxic effects from chemicals present in the particulates, trigger immunogenic responses leading to adverse reactions or interfere with the wound healing response, or interfere with the reabsorption of cerebrospinal fluid (CSF).
- Perform inspection before use according to "10 Check before Operation". If abnormal vibration or noise is found, the following issues are suspected:
  - The internal consumables (ball bearings, etc.) are worn or damaged.
  - The chuck is open or there is a bad connection of each part.
  - System defect due to damaged/deformed Cutting Accessory.Using the attachment in these conditions may lead to heat generation causing thermal injury. Perform "13 Troubleshooting" in this case. If the issue does not improve after troubleshooting, contact your Authorized NSK Dealer.
- DO NOT place the attachment/motor (including motor cord) on the patient or the drape covering the patient. Doing so may cause an accident, injury or thermal injury to the patient and/or surgical staff.
- Prevent any debris or other foreign materials from entering into the attachment. Failure to do so may lead to heat generation, causing thermal injury.
- DO NOT allow strong impact on the product for example, dropping the attachment/motor on a hard surface. Doing so may cause personal injury, abnormal heating, fire and/or thermal injury due to product damage.

## WARNING

- This attachment is a non-sterile product. After purchase or repair of the attachment, be sure to clean, lubricate and sterilize the attachment according to this operation manual before use (See to "12 Reprocessing").
- DO NOT use bent, damaged or deformed Cutting Accessories. If they are used, the attachment may not clamp the Cutting Accessory securely and it may break or cause accidents.
- DO NOT put excessive pressure on the Cutting Accessory. Doing so may bend or damage the Cutting Accessory, or cause excessive heat generation on the Cutting Accessory, attachment and motor, which could lead to thermal injury.
- Avoid continuous use of the product and use it intermittently to prevent excessive heat generation.  
In case of heat generation, immediately stop use until the product cools down.  
Be careful not to touch the heated area. Doing so may cause thermal injury.
- When replacing the Cutting Accessory, be sure to slide the safety lock on the hand switch of the motor to the OFF position, or completely take your foot off the foot control pedal to stop the attachment completely.  
Otherwise, a patient or surgical staff may be injured by unintentional movement of the motor and the Cutting Accessory.
- DO NOT operate the attachment if the markings of the twist collet are not aligned (chuck is open) after the Cutting Accessory is mounted. Doing so may cause thermal injury due to heat generation and potentially damage to the attachment.
- Be sure to adjust the exposure length of the Cutting Accessory outside the operative field and confirm that the exposure length is adjusted as intended before use.
- DO NOT use the Primado2 control unit and motor in a room with electromagnetic interference or adjacent to portable/mobile RF communication equipment. Use in such environments may affect the control unit and the motor. When using an ultrasonic generator or electrosurgical cutting and coagulation device that produces electromagnetic waves around them, turn off the power of the control unit.

**CAUTION**

- U.S. Federal law restricts these attachments to sale by or on the order of a licensed physician.
- Users shall be responsible for any judgement that relates to the application of this product to a patient.
- Users are responsible for the operational control, maintenance and continual inspection of this system.
- Be sure to read this operation manual and the operation manuals for "Total Surgical System Primado2", "Total Surgical System Primado2 Motor (P200-SMH/P200-SMH-HS/P200-SMH-S parts)" in order to become completely familiar with the function of each part before use.
- Be sure to perform "10 Check before Operation" by operating the product with the motor, attachments and Cutting Accessory mounted before use. If any abnormal condition is detected, stop using the product and perform "13 Troubleshooting".
- Wear Personal Protective Equipment (PPE) for safety and health when using the Cutting Accessories.
- Use the Cutting Accessories specified by NSK only. Using other manufacturer's Cutting Accessories may cause a fault or an accident.
- It is recommended that spare product be kept on hand in case of a breakdown during surgery.
- Never turn the twist collet during operation of the attachment.
- DO NOT touch the rotating part of any attachments during operation.

## CAUTION

- If the rotating Cutting Accessory catches hold of operating table drapes, injury and/or damage to the Cutting Accessory and the system may occur. DO NOT allow the rotating Cutting Accessory to come in contact with drapes.
- Make sure that the shank part of the Cutting Accessories to be attached to the attachment is clean. An unclean shank may cause center deviation or prevent proper clamping.
- After mounting the Cutting Accessory and locking the chuck, pull it lightly to confirm that it is securely connected to the attachment.
- DO NOT operate the attachment when the Cutting Accessory is not mounted.
- Before removing the Cutting Accessory from the attachment, stop the rotation of the attachment completely.
- Be sure to slide the safety lock on the hand switch of the motor to the OFF position when mounting or removing the attachment (when using P200-SMH-HS).
- The Irrigation Nozzle and the hood/beak cannot be used at the same time on the attachment.
- When mounting or removing the motor and attachment while using Primado2 control unit and motor, be sure to turn off the main power switch of the control unit. If the main power switch is ON, an unintentional movement may occur, which could result in injury to users (when using P200-SMH/P200-SMH-HS/P200-SMH-S).
- Use new Cutting Accessories for every surgical operation for safety.
- This device is for indoor use only.

**CAUTION**

- Perform regular function and maintenance checks for the devices and parts (See to "11 Periodical Maintenance Checks").
- If the product is not used for a long period of time, check if it functions correctly and safely before use.
- If chemicals, solutions or disinfectants are left on the surface of the attachment, wipe it away immediately. If it is left, it may lead to discoloration, deformation, rust and/or corrosion.
- DO NOT use the following fluids to wipe, immerse or clean the product: strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, and solvents such as benzine or thinner.
- The temperature at the tip of the attachment may reach up to 60°C (140°F) depending on the environment.
- Check the correct connection of the followings before using Primado2 Control Unit.
  - Connection of the attachment and motor.
  - Connection of the motor cord Plug and foot control cord plug to the control unit.
  - Connection of the power cord connector to the control unit.
  - Connection of the power cord plug to the outlet.

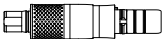
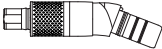
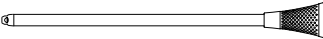
### 3 Package Contents

#### ◇ Standard Attachment 300 (Specification: see to "4 Part Names and Specifications")

Product Name	Order Code	Quantity	Fig.
STD Attachment 10/Straight	P300-2S10	1	
STD Attachment 20/Straight	P300-2S20	1	
STD Attachment 40/Straight	P300-2S40	1	
STD Attachment 60/Straight	P300-2S60	1	
STD Attachment 80/Straight	P300-2S80	1	
STD Attachment 10/Angled	P300-2A10	1	
STD Attachment 20/Angled	P300-2A20	1	
STD Attachment 40/Angled	P300-2A40	1	
STD Attachment 60/Angled	P300-2A60	1	
STD Attachment 80/Angled	P300-2A80	1	
Operation Manual	-	1	-

\*One of the product is included in the package.

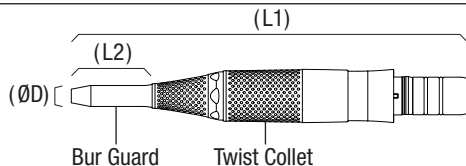
◇ **Slim Attachment 300 (Specification: see to "4 Part Names and Specifications")**

Product Name	Order Code	Quantity	Fig.
Slim Attachment Hub/Straight	P300-1AHS	1	
Slim Attachment Hub/Angled	P300-1AHA	1	
Slim Tube 110	P300-1T110	1	
Slim Tube 130	P300-1T130	1	
Slim Tube 170	P300-1T170	1	
Slim Tube 240	P300-1T240	1	
Slim Tube 110/C	P300-1T110-C	1	
Slim Tube 130/C	P300-1T130-C	1	
Slim Tube 110/PB	P300-1T110-PB	1	
Slim Tube 130/PB	P300-1T130-PB	1	
Operation Manual	-	1	-

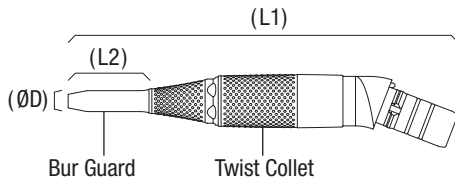
\*One of the product is included in the package.

## 4 Part Names and Specifications

### ◇ Standard Attachment 300

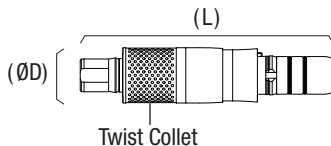
Order Code	P300-2S10	P300-2S20	P300-2S40	P300-2S60	P300-2S80
Maximum Rotation Apeed	80,000min <sup>-1</sup>				
Adjustable range of Cutting Accessories (Variable)	0 - 12.5mm				
Dimension of Attachment (L1)	89.0mm	104.0mm	124.0mm	144.0mm	164.0mm
Dimension of Bur Guard ( $\varnothing D$ x L2)	$\varnothing 5.5$ x 10.0mm	$\varnothing 5.5$ x 20.0mm	$\varnothing 5.5$ x 40.0mm	$\varnothing 5.5$ x 60.0mm	$\varnothing 5.5$ x 80.0mm
Applicable Beak	-		P300-2-B		
Applicable Irrigation Nozzle	PD-IN-D		PD-IN-2ES	PD-IN-2S	PD-IN-2M
Applicable Cutting Accessories	See P21 Table - 1.				

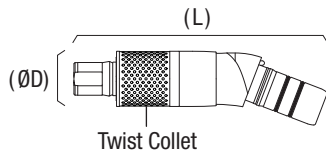
Order Code	P300-2A10	P300-2A20	P300-2A40	P300-2A60	P300-2A80
Maximum Rotation Speed	80,000min <sup>-1</sup>				
Adjustable range of Cutting Accessories (Variable)	0 - 12.5mm				
Dimension of Attachment (L1)	88.0mm	103.0mm	123.0mm	143.0mm	163.0mm
Dimension of Bur Guard (ØD x L2)	Ø5.5 x 10.0mm	Ø5.5 x 20.0mm	Ø5.5 x 40.0mm	Ø5.5 x 60.0mm	Ø5.5 x 80.0mm
Applicable Beak	-		P300-2-B		
Applicable Irrigation Nozzle	PD-IN-D		PD-IN-2ES	PD-IN-2S	PD-IN-2M
Applicable Cutting Accessories	See P21 Table - 1.				

## ◇ Slim Attachment 300

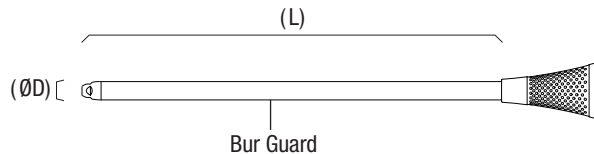
Order Code	P300-1AHS			
Maximum Rotation Speed	80,000min <sup>-1</sup>			
Adjustable range of Cutting Accessories (Variable)	0 - 10mm			
Applicable Slim Tube	P300-1T110 P300-1T110-C P300-1T110-PB	P300-1T130 P300-1T130-C P300-1T130-PB	P300-1T170	P300-1T240
Dimension (ØD x L)	Ø14.5 x 66.0mm			
Applicable Cutting Accessories	See P24 Table - 2.			

Order Code	P300-1AHA			
Maximum Rotation Speed	80,000min <sup>-1</sup>			
Adjustable range of Cutting Accessories (Variable)	0 - 10mm			
Applicable Slim Tube	P300-1T110 P300-1T110-C P300-1T110-PB	P300-1T130 P300-1T130-C P300-1T130-PB	P300-1T170	P300-1T240
Dimension (ØD x L)	Ø14.5 x 67.0mm			
Applicable Cutting Accessories	See P24 Table - 2.			

Order Code	P300-1T110 P300-1T110-C P300-1T110-PB	P300-1T130 P300-1T130-C P300-1T130-PB	P300-1T170*	P300-1T240*
Applicable Slim Attachment Hub	P300-1AHS / P300-1AHA			
Dimension of Bur Guard (ØD x L)	Ø4.8 x 110.0mm	Ø4.8 x 130.0mm	Ø4.8 x 170.0mm	Ø4.8 x 240.0mm
Applicable Hood/Beak	P300-1T-H / P300-1T-B			
Applicable irrigation nozzle	P200-IN-1A155	P200-IN-1A175	-	

\* These products can be used for endoscopic spine surgery under saline irrigation.

P300 Attachment Compatible Primado2 Products

<Control Unit>

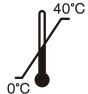
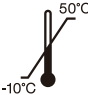
Product Name	Primado2 Control Unit (120V)
Order Code	P200-CU-120

<Foot Control>

Product Name	Primado2 Multi Foot Control	Primado2 Single Foot Control
Order Code	FC-74	FC-73

<Motor>

Product Name	Primado2 Slim Motor	Primado2 Slim Motor/HS	Primado2 Slim Motor Short
Order Code	P200-SMH	P200-SMH-HS	P200-SMH-S

Use environment			-
Store and transportation environment			

## 5 Mounting and Removing the Slim Tube

### ◇ Slim Attachment 300

#### CAUTION

Be sure to remove the Cutting Accessories from the attachment before removing the Slim Tube to prevent injury to the user or damage to the product.

#### <Mounting>

Insert the Slim Tube straight into the Slim Attachment Hub while rotating it slightly with fingers, then push the Slim Tube in until it clicks (Fig.1).

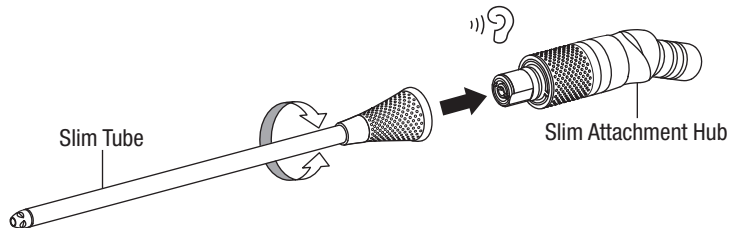
Fig.1

#### <Removing>

Pull the Slim Tube straight out from the Slim Attachment Hub.

## 6 Mounting and Removing the Attachment

### WARNING

- Be sure to slide the safety lock on the hand switch of the motor to the OFF position when mounting or removing the attachment (when using the motor with hand switch).
- See to the operation manual of Total Surgical System Primado2 Motor.
- Remove the Cutting Accessory before mounting or removing the attachment to prevent personal injury.
- After mounting the attachment, make sure that it is correctly connected to the motor.

#### <Mounting>

- 1 Insert the attachment straight into the motor.
- 2 Align the '▶' mark of the attachment and '●' mark of the motor, then push the attachment straight in until it clicks to connect firmly (Fig.2).

#### <Removing>

Pull the attachment straight out from the motor.

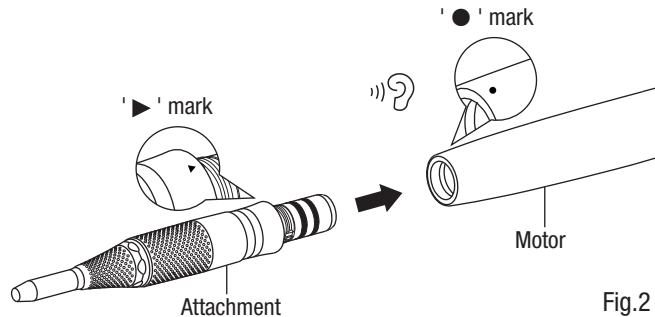
Fig.2

## 7 Mounting and Removing the Cutting Accessories

### WARNING

- Be sure to use the NSK recommended Cutting Accessory for each Attachment (See to "4 Part Names and Specifications: Applicable Cutting Accessories"). When using an unrecommended Cutting Accessory, it cannot be firmly fixed. This may lead to the Cutting Accessory coming off during surgical operation.
- DO NOT rotate the attachment if the chuck of the attachment is open. Doing so may cause thermal injury and/or damage to the product due to heat generation of the attachment.

### CAUTION

- After mounting the Cutting Accessory and locking the chuck, pull it lightly to confirm that it is securely connected to the attachment.
- Before use, confirm that the Cutting Accessory does not come off by operating the attachment.
- DO NOT operate the attachment when the Cutting Accessory is not mounted.
- Before mounting or removing the Cutting Accessory, be sure to stop the attachment completely.
- Be careful not to accidentally open the chuck while adjusting the exposure length of the Cutting Accessory. The Cutting Accessory can drop out when the chuck opens (when using Standard Attachment 300).
- The exposure length of the Cutting Accessory cannot be adjusted if it has only one identification line.

## ◇ Standard Attachment 300

To confirm the compatible Cutting Accessory for the Standard Attachment 300, see to Table - 1.

Table - 1

Compatible Cutting Accessory
PDS-2 □ □ UL- □ □

### <Mounting>

- 1 Pull the twist collet in the direction of **(A)** while rotating it in the direction of **(B)** until it clicks.  
Insert the Cutting Accessory, while twisting it slightly with your fingers (Fig.3).

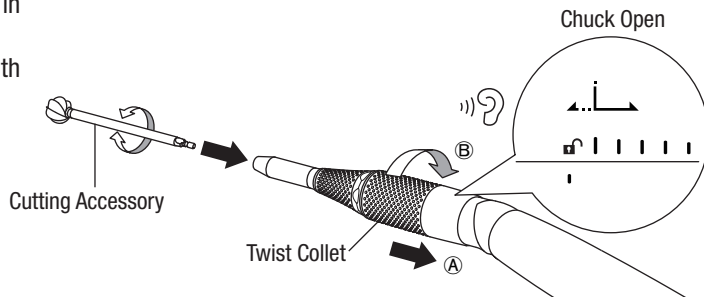
Fig.3

- 2** Rotate the twist collet in the direction of © until it clicks to fix the Cutting Accessory (Fig.4).

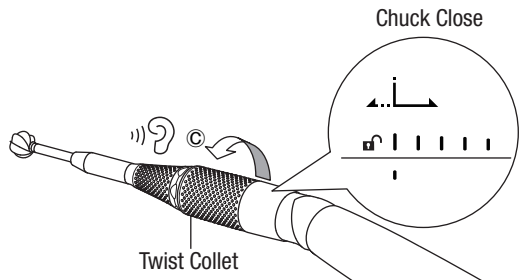
Fig.4

- 3** Pull the Cutting Accessory lightly to confirm that it is securely connected to the attachment (Fig.5).

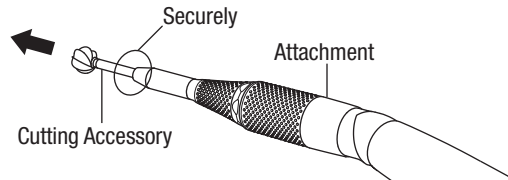
Fig.5

### <Adjustment of the exposure length of the Cutting Accessories>

The exposure length of the Cutting Accessory can be adjusted by rotating the twist collet of the attachment in the direction of ② or ③ while pulling it lightly in the direction of ①. See to the figure below (Fig.6).

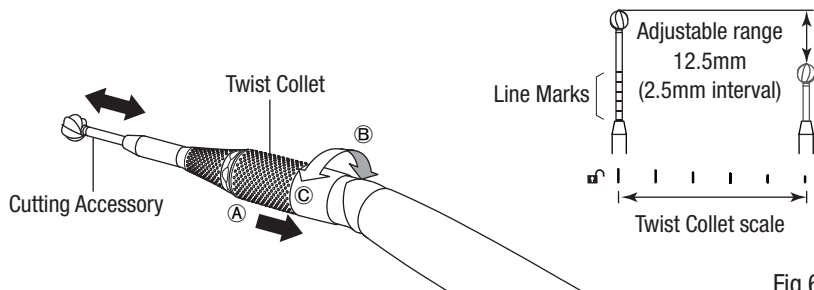
Fig.6

### <Removing>

Perform steps 1 and 2 of the mounting procedure in the reverse order.

## ◇ Slim Attachment 300

To confirm the compatible Cutting Accessory to Slim Attachment 300, see to Table - 2.

Table - 2

Slim Attachment Hub	Slim Tube	Compatible Cutting Accessory
P300-1AHA P300-1AHS	P300-1T110	PDS-1 □□ 110-□□
	P300-1T110-C	
	P300-1T110-PB	
	P300-1T130	PDS-1 □□ 130-□□
P300-1T130-C		
P300-1T130-PB		
	P300-1T170	PDS-1 □□ 170-□□
	P300-1T240	PDS-1 □□ 240-□□

### <Mounting>

- 1 Turn the twist collet in the direction of Ⓐ (Chuck open) until it clicks. Insert the Cutting Accessory all the way while twisting the Cutting Accessory slightly with your fingers (Fig.7).

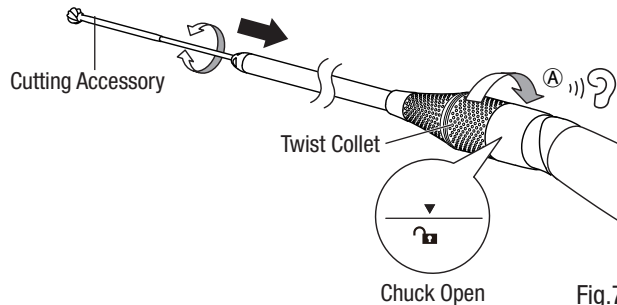
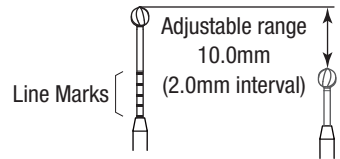
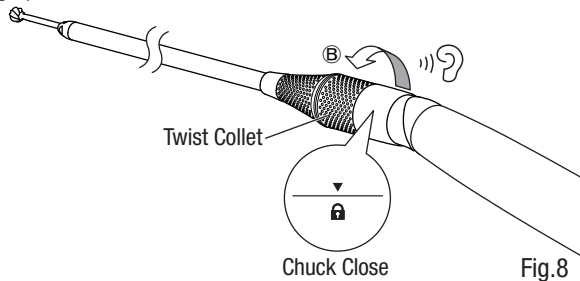
Fig.7

**2** Push or pull the Cutting Accessory to adjust the exposure length as required.

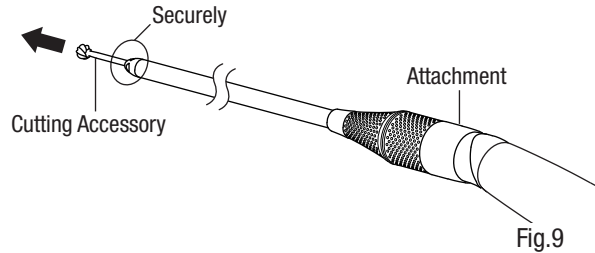
**NOTICE** • For Slim Attachment 300 compatible Cutting Accessory, the exposure length of its tip can be adjusted within the chucking range by changing the chucking position. The exposure variable range of the Cutting Accessory is 10mm.

**3** Rotate the twist collet in the direction of **Ⓑ** until it clicks. (Fig.8)

**4** Pull the Cutting Accessory lightly to confirm that it is securely connected to the attachment (Fig.9).

< Removing >

Perform steps 1 through 3 of the mounting procedure in the reverse order.

## 8 Mounting the Irrigation Nozzle (Optional)

### CAUTION

- Mount the Cutting Accessory to the attachment after mounting the irrigation nozzle.
- Be sure to use the NSK-specified irrigation nozzle (See to "4 Part Names and Specifications").
- See to the operation manual of Total Surgical System Primado2.

- 1 Insert the irrigation nozzle onto the bur guard of the attachment, making sure that the correct end of the irrigation nozzle goes in (Fig.10).
- 2 Insert the irrigation tube into the irrigation nozzle (Fig.10).
- 3 For the related connection and the setting on the control unit, see to the operation manual of Total Surgical System Primado2.

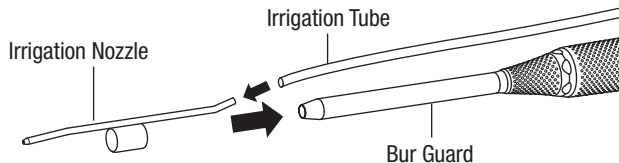
Fig.10

## 9 Mounting and Removing the Hood/Beak (Optional)

 **CAUTION**

- Be sure to mount the hood/beak after the Cutting Accessory is mounted on the attachment.
- Insert the hood/beak until '▶' mark on the bur guard is hidden when mounting the hood/beak.
- Use the following Cutting Accessory when mounting the hood/beak.

Standard Attachment 300	Tip diameter is Ø4mm or less
Slim Attachment 300	Tip diameter is Ø3mm or less

- \* The hood/beaks cannot be used for attachments other than the P300 Attachments.
- Be sure to completely stop rotation of the attachment before mounting/removing the hood/beak.

- 1 Make sure that the rotation of the attachment is stopped.
- 2 Mount the hood/beak on the Bur Guard (Fig.11).

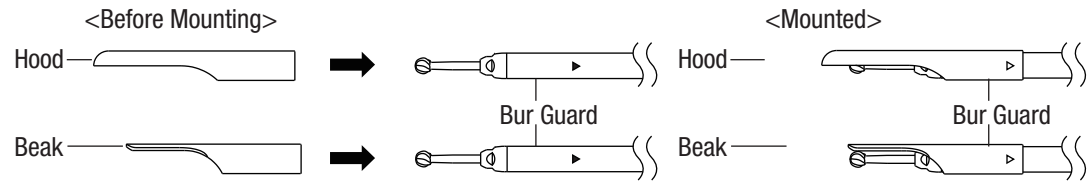
Fig.11

## 10 Check before Operation

Conduct check before operation according to the following procedure to confirm that there are no abnormalities. If any abnormality is observed, perform "13 Troubleshooting".

- 1 Attach the attachment to the motor, then attach the Cutting Accessory to the attachment.  
(See "6 Mounting and Removing the Attachment", "7 Mounting and Removing the Cutting Accessories (Optional)".)
- 2 Check for abnormalities at the connection part of the attachment and Cutting Accessory.
- 3 Operate the attachment at the rotational speed permissible for the Cutting Accessory mounted. Confirm that nothing is loose and that no abnormal vibration, heating or noise occurs.

If any abnormality is observed, stop use and perform "13 Troubleshooting".

- 4 Ensure that there are no abnormalities before use.

## 11 Periodical Maintenance Checks

Every three months, perform periodical maintenance checks on this product according to the following list. If any abnormalities are found, contact your Authorized NSK Dealer.

Checking Item	Description
Outer Surface	Check for any scratches, damage or abnormalities on the outer surface.
Each Connection Part	Check for any scratches, damage or deformation of each connection part.
Attachment	Connect the attachment and check for abnormalities at the connection part.
Rotation	After connecting each part, fasten the Twist Collet and rotate the motor for approximately 1 minute by pressing the foot control pedal or Hand Switch of the motor. Check for any abnormal vibration, noise or heat generation. Check that the motor stops normally after releasing the foot control pedal or Hand Switch of the motor.

## 12 Reprocessing

The reprocessing instructions are created based on the requirements of ISO 17664.

The effectiveness of the following reprocessing procedures have been verified.

### WARNING

- Since no reprocessing methods have been validated for removal of transmissible spongiform encephalopathy (TSE) agents from medical devices, the reusable devices should not be used for patients with known or suspected Creutzfeldt-Jakob Disease(CJD). In case of using this device on a patient infected by Creutzfeldt-Jakob disease or when contamination with high risk tissues/organs is suspected, the devices shall be disposed immediately.
- Follow the regulations or guidelines of each country.
- Reprocess this product after every surgical operation.
- The Cutting Accessory is for single use only. DO NOT reuse or reprocess it. If you reuse or reprocess it, the following risks may occur:
  - Infection.
  - Personal injury due to damaged Cutting Accessory
  - System defect due to damaged/deformed Cutting Accessory.

**WARNING**

- Thoroughly clean the attachment before autoclave sterilization. Autoclaving the attachment with blood remaining on it may lead to sterilization failure, resulting in infection to the patient and/or user or malfunction of the attachment due to blood coagulation.
- Immediately after use, the attachment should be cleaned, lubricated and sterilized. Remaining blood in the attachment may cause rust due to blood coagulation. Failure to properly maintain the attachment may lead to overheating, causing thermal injuries or product failure.

**CAUTION**

- For detailed information on cleaning and sterilizing the motor, see to the operation manual of Total Surgical System Primado2 Motor.
- Wear Personal Protective Equipment (PPE) to prevent infection.
- NSK has verified the effectiveness of reprocessing using the dedicated cleaning tools. Be sure to use the dedicated cleaning tools for reprocessing.

## 12-1 Preparation

### Treatment at the Operating Site

- 1 Remove visible contaminants such as blood using a lint-free cloth moistened with distilled water, deionized water or mineral-free water at the operating site.
- 2 Remove the Cutting Accessory, motor, irrigation nozzle, hood/beak and other connections from the attachment (For removing the attachment from the motor, see to the operation manual of Total Surgical System Primado2 Motor).
- 3 Remove the Slim Tube from the Slim Attachment Hub (when using Slim Attachment 300).

## 12-2 Cleaning

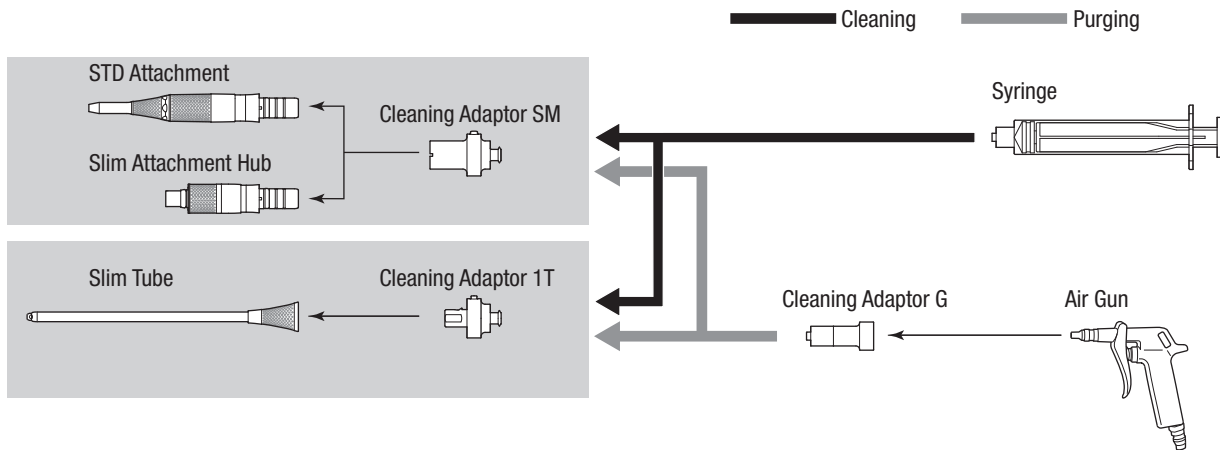
### CAUTION

- DO NOT use a lint cloth or metal brush for cleaning.
- DO NOT use a chlorinated cleaner, benzene or thinner for cleaning to prevent damage to the product.
- DO NOT clean the product with an ultrasonic cleaning apparatus to prevent damage to the product.
- When using detergents other than the ones we used for validation, it is necessary to verify the effectiveness based on the information provided by the manufacturer of the detergent and the information obtained from testing.
- Prepare the detergent according to the instructions of the detergent manufacturer when using the detergent.
- See to the operation manual of the washer disinfecter before operating it.
- Be sure to use a disinfecter compliant with ISO15883.

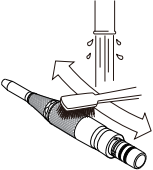
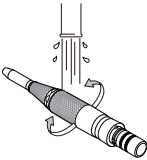
## 12-2-1 Manual Cleaning

### ◇ Standard Attachment 300, Slim Attachment 300

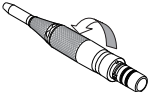
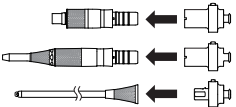
Use the cleaning tools (Optional) according to the following system figure.

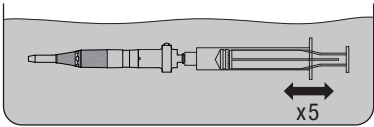
## Pre-cleaning

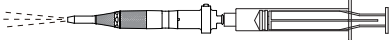
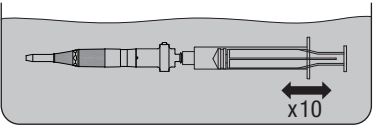
No.	Procedure	Fig.
<b>1</b>	<p>Remove contaminants on the surface sufficiently with a brush under running tap water.</p> <p>For products with the twist collet, perform cleaning while moving the twist collet.</p> <p>≤ 38° C (104° F), ≥ 3.5L/min, ≥ 30 sec.</p> <p>Water quality: Drinking water</p> <p>* For Slim Attachment 300, perform this step with the Slim Tube removed from the Slim Attachment Hub.</p>	<b>1</b>  

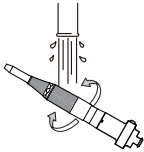
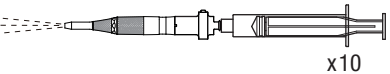
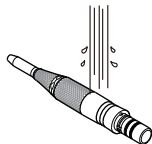
## Cleaning

<b>1</b>	Turn the twist collet to the close position.	<b>1</b> 
<b>2</b>	Mount the cleaning adaptor. Pull the cleaning adaptor to make sure that it is securely connected.	<b>2</b> 

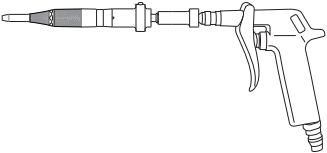
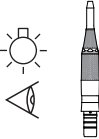
No.	Procedure	Fig.
<b>3</b>	Screw the disposable luer-lock syringe into each cleaning adaptor of the attachment.	<b>3</b> 
<b>4</b>	Prepare the cleaning solution in the cleaning tank under the following conditions: (Prepare the cleaning solution according to the instructions of the detergent manufacturer).	
	Temperature	Detergent
45° C (113°F)	Water quality	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)
	Distilled, deionized, or demineralized water	
*Detergent used for validation: Getinge Clean MIS Detergent (GETINGE)		
<b>5</b>	With the product immersed in the cleaning tank, pull the syringe back, then pump the syringe to suck and discharge the cleaning solution. Perform this step 5 times to retain the cleaning solution inside the product. Then, immerse the product in the cleaning tank for 20-30 min.	<b>5</b> 

## Cleaning (Continued)

No.	Procedure	Fig.
<b>6</b>	Push and pull the syringe pusher to discharge the cleaning solution remaining in the product.	<b>6</b> 
<b>7</b>	Immerse the product in the soak tank containing tap water, then operate the syringe to suck and discharge water 10 times to rinse inside the product. ≤ 38°C (104°F) Water quality: Drinking water	<b>7</b> 

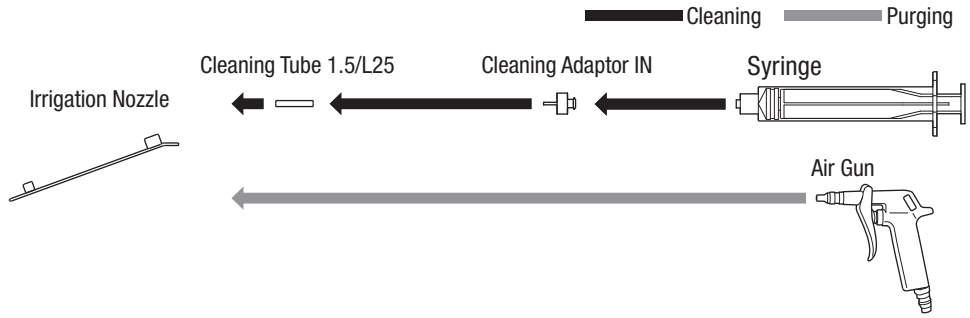
No.	Procedure	Fig.
<b>8</b>	Rinse the product while under running tap water. For products with the twist collet, perform rinsing while moving the twist collet. $\leq 38^{\circ}\text{C}$ ( $104^{\circ}\text{F}$ ), $\geq 3.5\text{L/min}$ , $\geq 10$ sec. Water quality: Drinking water	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <p><b>8</b></p>  </div> <div style="text-align: center;"> <p><b>9</b></p>  </div> </div>
<b>9</b>	Put distilled, deionized or demineralized water into the syringe, screw the syringe into the product, and discharge water from the product tip. Perform this step 10 times.	<p><b>10</b></p> 
<b>10</b>	Rinse the product surface with distilled, deionized or demineralized water.	

## Purging

No.	Procedure	Fig.					
<b>1</b>	Screw the Cleaning Adaptor G into each cleaning adaptor.	<b>1</b>	<b>2</b>				
<b>2</b>	<p>Push the air blow gun into the Cleaning Adaptor G and remove moisture inside the product.</p> <table border="1" data-bbox="308 410 1111 550"> <tr> <td data-bbox="308 410 671 477">Allowable air pressure</td> <td data-bbox="671 410 1111 477">&lt; 500 kPa</td> </tr> <tr> <td data-bbox="308 477 671 550">Purging time</td> <td data-bbox="671 477 1111 550">≥ 5sec.</td> </tr> </table> <p>Remove moisture on the surface of the product using a dry cloth, air gun, etc.</p>	Allowable air pressure	< 500 kPa	Purging time	≥ 5sec.		
Allowable air pressure	< 500 kPa						
Purging time	≥ 5sec.						
<b>3</b>	Remove the cleaning adaptor from the attachment by depressing the release button.	<b>3</b>	<b>4</b>				
<b>4</b>	Visually check that all contamination is removed. In case there is residual contamination, repeat the procedures of Pre-cleaning through Purging until all contamination is removed.						

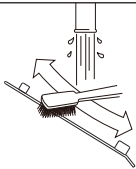
◇ Irrigation Nozzle (Optional)

Use the Cleaning Tools (optional) according to the following system figure.

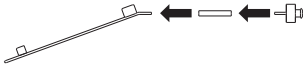
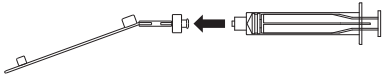
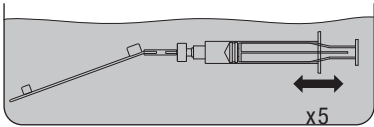
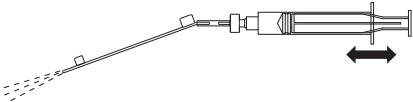
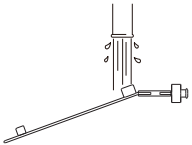
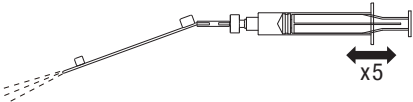
**Pre-cleaning**

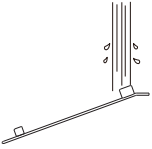
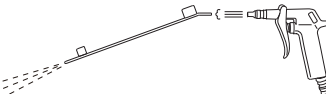
No.	Procedure	Fig.
<b>1</b>	Remove contaminants on the surface with a brush under running tap water. ≤ 38°C (104°F), ≥ 3.5L/min, ≥ 15 sec. Water quality: Drinking water	<b>1</b> 

**Cleaning**

<b>1</b>	Mount the cleaning adaptor to the Cleaning Tube 1.5/L25. Pull the cleaning adaptor to make sure that it is securely connected. * If the Cleaning Tube 1.5/L25 is deteriorated, replace it with a new one.	<b>1</b> 
<b>2</b>	Screw the disposable luer-lock syringe into the cleaning adaptor IN.	<b>2</b> 

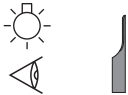
No.	Procedure	Fig.						
<b>3</b>	<p>Prepare the cleaning solution in the cleaning tank under the following conditions: (prepare the cleaning solution according to the instructions of the detergent manufacturer).</p> <table border="1" data-bbox="312 277 2161 417"> <thead> <tr> <th data-bbox="312 277 562 350">Temperature</th> <th data-bbox="562 277 1208 350">Water quality</th> <th data-bbox="1208 277 2161 350">Detergent</th> </tr> </thead> <tbody> <tr> <td data-bbox="312 350 562 417">45° C (113° F)</td> <td data-bbox="562 350 1208 417">Distilled, deionized, or demineralized water</td> <td data-bbox="1208 350 2161 417">Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)</td> </tr> </tbody> </table> <p>*Detergent used for validation: Getinge Clean MIS Detergent (GETINGE)</p>	Temperature	Water quality	Detergent	45° C (113° F)	Distilled, deionized, or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)	
Temperature	Water quality	Detergent						
45° C (113° F)	Distilled, deionized, or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)						
<b>4</b>	<p>With the product immersed in the cleaning tank, pull the syringe back, then pump the syringe to suck and discharge the cleaning solution. Perform this step 5 times to retain the cleaning solution inside the product. Then, immerse the product in the cleaning tank for 20-30 min.</p>	<p><b>4</b></p> 						

Cleaning (Continued)		
No.	Procedure	Fig.
5	Push and pull the syringe plunger to discharge the cleaning solution remaining in the product.	5 
6	Rinse the product under running tap water. $\leq 38^{\circ}\text{C}$ (104°F), $\geq 3.5\text{L/min}$ , $\geq 10$ sec. Water quality: Drinking water	6 
7	Put distilled, deionized or demineralized water into the syringe, screw the syringe into the product, and discharge water from the product tip. Perform this step 5 times.	7 

No.	Procedure	Fig.
<b>8</b>	Remove the Cleaning Tube 1.5/L25 and the Cleaning Adaptor IN from the Irrigation Nozzle.	<div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;"> <p><b>8</b></p>  </div> <div style="text-align: center;"> <p><b>9</b></p>  </div> </div>
<b>9</b>	Rinse the product surface with distilled, deionized or demineralized water.	
<b>Purging</b>		
<b>1</b>	Remove moisture inside and on the surface of the product using a dry cloth, air gun, etc.	<div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;"> <p><b>1</b></p>  </div> <div style="text-align: center;"> <p><b>2</b></p>  </div> <div style="text-align: center;">  </div> </div>
<b>2</b>	Visually check that all contamination is removed. In case there is residual contamination, repeat the procedures of Pre-cleaning through Purging until all contamination is removed.	

## ◇ Hood/Beak (Optional)

Pre-cleaning								
No.	Procedure	Fig.						
<b>1</b>	Remove contaminants on the surface with a brush under running tap water. $\leq 38^{\circ}\text{C}$ (104°F), $\geq 3.5\text{L/min}$ , $\geq 15\text{ sec.}$ Water quality: Drinking water	<b>1</b> 						
Cleaning								
<b>1</b>	Perform immersion cleaning under the following conditions: <table border="1" data-bbox="306 673 2154 813"> <thead> <tr> <th>Temperature</th> <th>Water quality</th> <th>Detergent</th> </tr> </thead> <tbody> <tr> <td>45°C (113°F)</td> <td>Distilled, deionized or demineralized water</td> <td>Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)</td> </tr> </tbody> </table> <p>* Detergent used for validation: Getinge Clean MIS Detergent (GETINGE)</p> <p>Immerse the product in the cleaning tank for 20-30 min.</p>		Temperature	Water quality	Detergent	45°C (113°F)	Distilled, deionized or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)
Temperature	Water quality	Detergent						
45°C (113°F)	Distilled, deionized or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)						

No.	Procedure	Fig.	
<b>2</b>	Rinse the product under running tap water. ≤ 38° C (104° F), ≥ 3.5L/min, ≥ 10 sec. Water quality: Drinking water	<b>2</b>	<b>3</b>
<b>3</b>	Rinse the product surface with distilled, deionized or demineralized water.		
<b>Purging</b>			
<b>1</b>	Remove moisture inside and on the surface of the product using a dry cloth, air gun, etc.	<b>1</b>	<b>2</b>
<b>2</b>	Visually check that all contamination is removed. In case there is residual contamination, repeat the procedures of Pre-cleaning through Purging until all contamination is removed.		

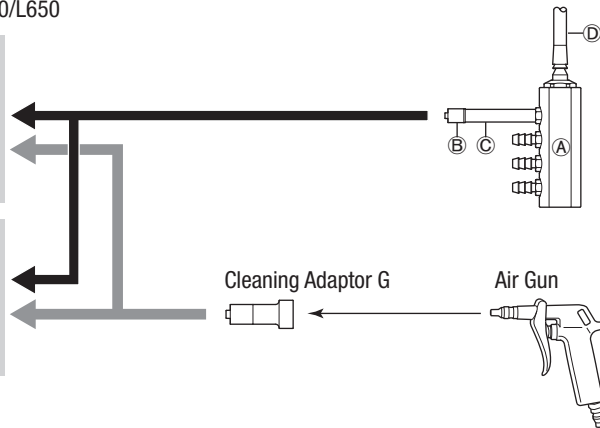
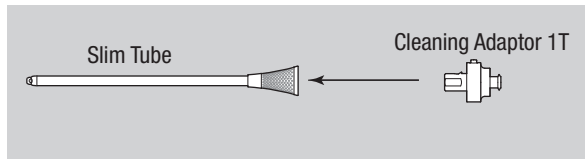
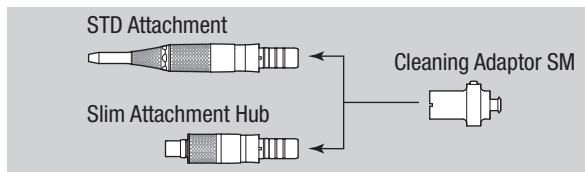
## 12-2-2 Automated Cleaning

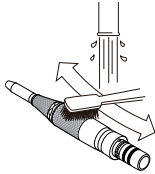
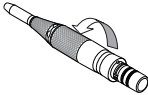
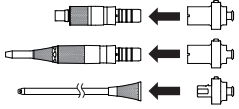
### ◇ Standard Attachment 300, Slim Attachment 300

Use the cleaning tools (Optional) according to the following system figure.

- Ⓐ Cleaning Holder   Ⓑ Cleaning Adaptor Luer M   Ⓒ Cleaning Tube 6.0/L50   Ⓓ Cleaning Tube 6.0/L650

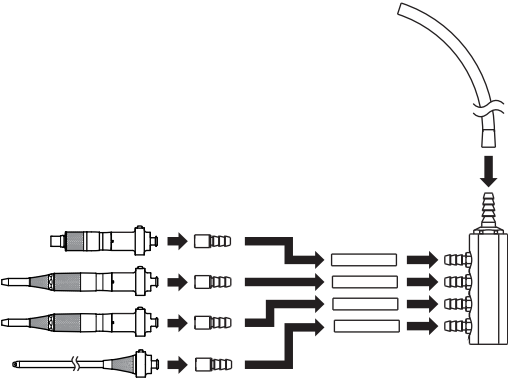
————— Cleaning   ——— Purging

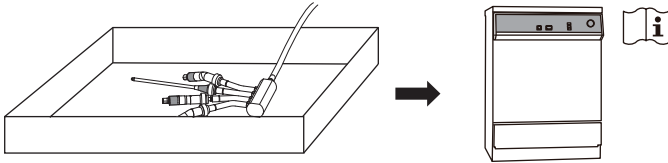
Pre-cleaning		
No.	Procedure	Fig.
<b>1</b>	<p>Remove contaminants on the surface with a brush under running tap water.</p> <p>For products with the twist collet, perform cleaning while moving the twist collet.</p> <p>≤ 38° C (104° F), ≥ 3.5L/min, ≥ 30 sec.</p> <p>Water quality: Drinking water</p> <p>* For Slim Attachment 300, perform this step with the Slim Tube removed from the Slim Attachment Hub.</p>	
<b>2</b>	Turn the twist collet to the close position.	
<b>3</b>	<p>Mount the cleaning adaptor.</p> <p>Pull the cleaning adaptor to make sure that it is securely connected.</p>	

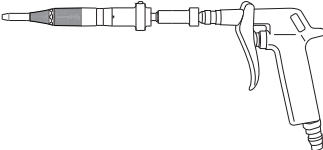
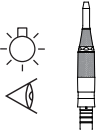
## Pre-cleaning (Continued)

No.	Procedure	Fig.
<b>4</b>	<p>Assemble the product as shown in the figure.</p> <ul style="list-style-type: none"><li>– Pull each connection part to make sure that it is securely connected.</li><li>* If the cleaning tube is deteriorated, replace it with a new one.</li></ul>	<b>4</b> 

No.	Procedure	Fig.
<b>5</b>	Place the Cleaning Holder in the cleaning basket and set the cleaning basket in the washer disinfectant.	<b>5</b> 
<b>6</b>	Mount the Cleaning Tube 6.0/L650 to the washer disinfectant.	

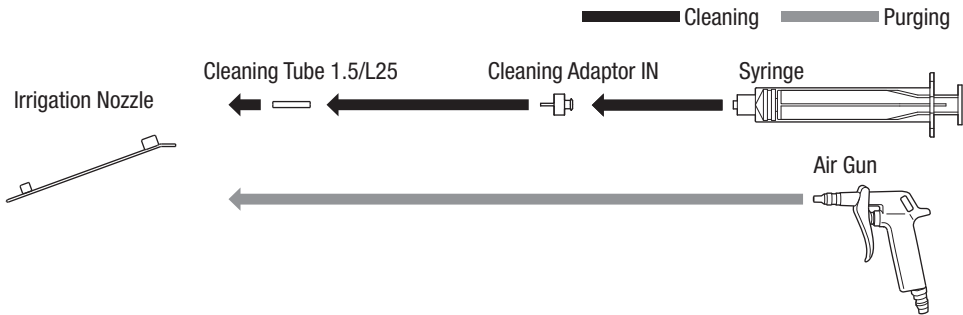
## Cleaning and Disinfection

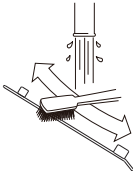
No.	Procedure				
<b>1</b>	Perform cleaning and thermal disinfection using the washer disinfector under the following conditions:				
		Temperature	Time	Water quality	Detergent
	Pre-cleaning	< 25° C (77° F)	3 min	Tap water	-
	Cleaning	40 - 60° C (104 - 140° F)	≥ 5 min	Distilled, deionized or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)
	Rinsing	> 10° C (50° F)	≥ 1 min		-
	Thermal disinfection	93° C (199° F)	5 min		-
	Drying	Set the washer disinfector according to the manufacturer's instruction.			
<p>* Detergent used for validation: Neutral enzymatic detergent: Getinge Clean MIS Detergent (GETINGE)</p>					

Purging		Fig.				
<b>No.</b>	<b>Procedure</b>					
<b>1</b>	Screw the Cleaning Adaptor G into each cleaning adaptor.	<b>1</b>				
<b>2</b>	<p>Push the air blow gun into Cleaning Adaptor G and remove moisture inside the product.</p> <table border="1"> <tr> <td>Allowable air pressure</td> <td>&lt; 500 kPa</td> </tr> <tr> <td>Purging time</td> <td>≥ 5sec.</td> </tr> </table> <p>Remove moisture on the surface of the product using a dry cloth, air gun, etc.</p>	Allowable air pressure	< 500 kPa	Purging time	≥ 5sec.	<b>2</b>  
Allowable air pressure	< 500 kPa					
Purging time	≥ 5sec.					
<b>3</b>	Remove the cleaning adaptor from the attachment by depressing the Release button.	<b>3</b>				
<b>4</b>	Visually check that all contamination is removed. In case there is residual contamination, repeat the procedures of Pre-cleaning through Purging until all contamination is removed.	<b>4</b>  				

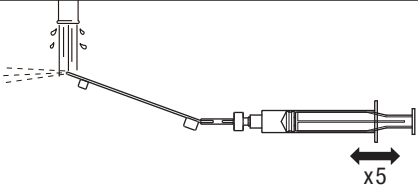
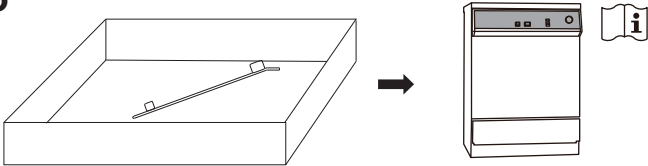
## ◇ Irrigation Nozzle (Optional)

Use the Cleaning Tools (Optional) according to the following system figure.

Pre-cleaning		
No.	Procedure	Fig.
<b>1</b>	Remove contaminants on the surface with a brush under running tap water. $\leq 38^{\circ}\text{C}$ ( $104^{\circ}\text{F}$ ), $\geq 3.5\text{L}/\text{min}$ , $\geq 15\text{ sec}$ . Water quality: Drinking water	<b>1</b> 
<b>2</b>	Mount the Cleaning Adaptor IN to the Cleaning Tube 1.5/L25. Pull the Cleaning Adaptor IN to make sure that it is securely connected. * If the Cleaning Tube 1.5/L25 is deteriorated, replace it with a new one.	<b>2</b> <b>3</b> 
<b>3</b>	Screw the disposable luer-lock syringe into the Cleaning Adaptor IN.	

## Pre-cleaning (Continued)

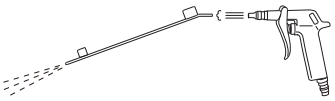
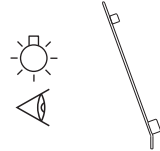
No.	Procedure	4 Fig.
<b>4</b>	Suck tap water from the product tip and discharge it. Perform this step 5 times. ≤ 38°C (104°F) Water quality: Drinking water	
<b>5</b>	Remove the Cleaning Tube 1.5/L25 and the Cleaning Adaptor IN from the Irrigation Nozzle.	
<b>6</b>	Place the irrigation nozzle in the cleaning basket and set the cleaning basket in the washer disinfector.	

## Cleaning and Disinfection

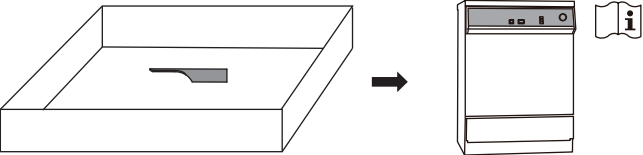
No.	Procedure				
<b>1</b>	Perform cleaning and thermal disinfection using the washer disinfectant under the following conditions:				
		Temperature	Time	Water quality	Detergent
	Pre-cleaning	< 25°C (77°F)	3 min	Tap water	-
	Cleaning	40 - 60°C (104 - 140°F)	≥ 5 min	Distilled, deionized or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)
	Rinsing	> 10°C (50°F)	≥ 1 min		-
	Thermal disinfection	93°C (199°F)	5 min		-
	Drying	Set the washer disinfectant according to the manufacturer's instruction.			
* Detergent used for validation: Getinge Clean MIS Detergent (GETINGE)					

**Purging**

No.	Procedure	Fig.
<b>1</b>	Remove moisture inside and on the surface of the product using a dry cloth, air gun, etc.	<b>1</b> <b>2</b> 
<b>2</b>	Visually check that all contamination is removed. In case there is residual contamination, repeat the procedures of Pre-cleaning through Purging until all contamination is removed.	

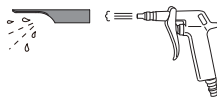
## ◇ Hood/Beak (Optional)

Pre-cleaning		
No.	Procedure	Fig.
<b>1</b>	<p>Remove contaminants on the surface with a brush under running tap water.</p> <p>≤ 38°C (104°F), ≥ 3.5L/min, ≥ 15 sec.</p> <p>Water quality: Drinking water</p>	<p><b>1</b></p> 
<b>2</b>	<p>Place the Hood/Beak in the cleaning basket and set the cleaning basket in the washer disinfectant.</p>	<p><b>2</b></p> 

## Cleaning and Disinfection

No.	Procedure				
<b>1</b>	Perform cleaning and thermal disinfection using the washer disinfector under the following conditions:				
		Temperature	Time	Water quality	Detergent
	Pre-cleaning	< 25°C (77°F)	3 min	Tap water	-
	Cleaning	40 - 60°C (104 - 140°F)	≥ 5 min	Distilled, deionized or demineralized water	Neutral enzymatic detergent for medical purposes (pH 7.0 - 8.0)
	Rinsing	> 10°C (50°F)	≥ 1 min		-
	Thermal disinfection	93°C (199°F)	5 min		-
	Drying	Set the washer disinfector according to the manufacturer's instruction.			
<p>* Detergent used for validation: Neutral enzymatic detergent: Getinge Clean MIS Detergent (GETINGE)</p>					

Purging		
No.	Procedure	Fig.
<b>1</b>	Remove moisture inside and on the surface of the product using a dry cloth, air gun, etc.	<b>1</b>
<b>2</b>	Visually check that all contamination is removed. In case there is residual contamination, repeat the procedures of Pre-cleaning through Purging until all contamination is removed.	<b>2</b>

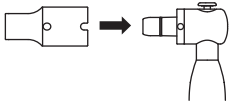
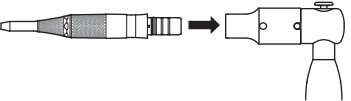
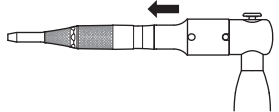
## 12-3 Maintenance

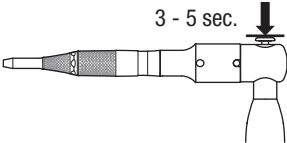
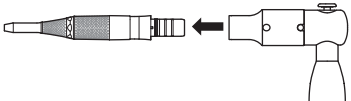
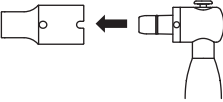
### CAUTION

- DO NOT use anything other than PANA SPRAY Plus for lubrication.
- Shake PANA SPRAY Plus well before use.
- Due to its structure, the lubrication oil of PANA SPRAY Plus comes out from the tip of the attachment and the twist collet part. To prevent dispersal of the lubrication oil in the surrounding area, it is recommended to cover the attachment with a tissue or use the Spray Mist Absorber.
- After lubrication and prior to sterilization, check the attachment in the same procedure of "10 Check before Operation".
- After lubrication, it could be high pressure inside the Spray Adaptor for Slim. When removing the Spray Adaptor for Slim, be careful because it may pop out with great force.

<b>Lubrication</b> Lubricate the product using separately sold EZ Spray Adaptor (Order Code PD-EZ-M) and PANA SPRAY Plus (Order Code Z182600).		
No.	Procedure	Fig.
<b>1</b>	Connect each part of PANA SPRAY Plus, EZ Spray Grip, and Spray Adaptor for Slim.	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <b>1</b>   </div> <div style="text-align: center;"> <b>2</b>   </div> </div>
<b>2</b>	Mount the Slim Tube to the Slim Attachment Hub. * When using the Slim Attachment 300.	
<b>3</b>	Mount the Attachment to the Spray Adaptor for Slim.	<div style="text-align: center;"> <b>3</b>   </div>
<b>4</b>	Pull the product to make sure that it is securely connected.	<div style="text-align: center;"> <b>4</b>   </div>

## Lubrication (Continued)

No.	Procedure	Fig.
<b>5</b>	Hold the Attachment firmly to prevent it from moving by the pressure of PANA SPRAY Plus. Spray PANA SPRAY Plus until the oil coming out from the end of the attachment becomes clear (3 - 5 seconds).	<p><b>5</b></p> 
<b>6</b>	Remove the attachment from the Spray Adaptor for Slim by depressing the Release button on the Spray Adaptor for Slim.	<p><b>6</b></p> 
<b>7</b>	Remove the Spray Adaptor for Slim from the EZ Spray Grip by depressing the release button on the EZ Spray Grip.	<p><b>7</b></p> 

Checking	
No.	Procedure
<b>1</b>	Confirm that there is no damage on the outer surface and connection parts of the product or missing parts.
<b>2</b>	Check with the same procedure described in "10 Check before Operation".

## 12-4 Packaging / Sterilization

We confirmed that the following sterile procedure is in accordance with ISO 17665.

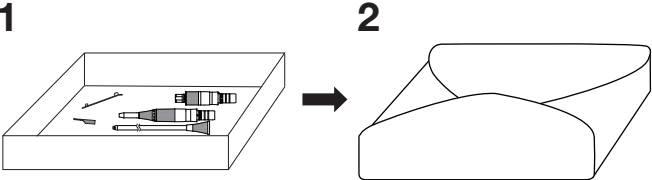
### CAUTION

- The sterilization parameters are indicated for sterilization and immediate use only.
- Use a sterilization cassette approved as a medical device (Use an FDA-approved sterilization cassette to perform sterilization).
- Use a wrap compliant with ISO 11607 (Use an FDA-approved wrap to perform sterilization).
- Follow the wrap manufacturer's instruction for how to use the wrap.
- DO NOT heat or cool the product too quickly when autoclaving. Rapid change in temperature could cause damage to the product.
- The validity of sterilization methods other than autoclaving is not confirmed.
- DO NOT touch the product immediately after autoclaving as the product is very hot.
- Use an FDA-approved steam sterilizer to perform sterilization.

## Packaging

No.	Procedure	Fig.
<b>1</b>	Store the product in a metal sterilization cassette. * For Slim Attachment 300, perform this step with the Slim Tube removed from the Slim Attachment Hub.	 <p>The figure consists of two parts labeled 1 and 2. Part 1 shows a rectangular metal sterilization cassette containing several tools: a pair of tweezers, a scalpel, a spray adaptor, and a slim attachment. Part 2 shows the same cassette wrapped in a piece of sterilization wrap, which is folded over the top and sides to completely enclose the cassette. An arrow points from part 1 to part 2.</p>
<b>2</b>	Wrap the sterilization cassette with sterilization wrap.	

## Sterilization

<b>1</b>	Perform autoclave sterilization under the following conditions: <Non Autoclavable Product> Spray Adaptor for Slim, Cleaning tools			
	Sterilization method	Sterilization temperature	Sterilization time	Drying time
	Pre-vacuum autoclave	132°C (269.6°F)	≥ 4 min	≥ 20 min

## 12-5 Storage

### CAUTION

- Store the products in a place avoiding adverse effects from atmospheric pressure, temperature, humidity, ventilation, direct sun, dust and salinity/sulfur content atmosphere.

Store the sterilized product in a place where it can be kept sterile until the next use.

## 12-6 Carrying to the Operating Site

Keep the Attachment sterile when it is carried to the operating site.

## 13 Troubleshooting

If you are suspect any faults, check the following troubleshooting guide prior to asking for repair work. If none of these cases is applicable or if the situation does not improve despite your efforts or measures, the damage could be serious; call your Authorized NSK Dealer to fix the problem.

Symptoms	Causes/Points of checking	Countermeasures
The attachment cannot be connected to the motor.	The ' ► ' mark of the attachment and ' ● ' mark of the motor are not aligned.	Align the ' ► ' mark of the attachment and ' ● ' mark of the motor, then push the attachment in until it clicks to connect firmly.
	Connection part has been deformed.	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be firmly mounted.	The Cutting Accessory has not been inserted deep enough.	
	The twist collet is in ' OPEN ' (Chuck Open position).	After inserting the Cutting Accessory, fasten the twist collet (Chuck close position).
	twist collet has been damaged or deformed.	Contact your Authorized NSK Dealer.
The exposure length of the Cutting Accessory cannot be adjusted.		

Symptoms	Causes/Points of checking	Countermeasures
No rotation when the Cutting Accessory is securely mounted.	<ul style="list-style-type: none"> <li>• The bearing has overheated.</li> <li>• Foreign materials inside.</li> <li>• The bearing has worn out.</li> </ul>	Contact your Authorized NSK Dealer.
Overheating (during operation).	The twist collet is in ' OPEN ' (Chuck Open position).	After inserting the Cutting Accessory, fasten the Twist Collet.
	The ' ► ' mark of the attachment and ' ● ' mark of the Motor are not aligned.	Align the ' ► ' mark of the attachment and ' ● ' mark of the motor, then push the attachment in until it clicks to connect firmly.
	Continuous use over a long period.	Stop using the attachment and the motor immediately. Wait until the attachment and the motor cool down before using the Attachment and the Motor again.
	<ul style="list-style-type: none"> <li>• The bearing has overheated.</li> <li>• Foreign materials inside.</li> <li>• The bearing has worn out.</li> </ul>	Contact your Authorized NSK Dealer.

Symptoms	Causes/Points of checking	Countermeasures
Abnormal vibration and noise. Cutting Accessory run-out.	<ul style="list-style-type: none"> <li>• The bearing has overheated.</li> <li>• Foreign materials inside.</li> <li>• The bearing has worn out.</li> </ul>	Contact your Authorized NSK Dealer.

## 14 Symbols

Manufacturer

State of being unfixed

Catalog number

Conforms to CE European Directive of “Medical device directive 93/42/EEC.”

Visual check

Serial No.

See operation manual

Lighting

Batch number

GS1 DataMatrix for Unique Device Identifier

Listen (Clicking sound)

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

Temperature limitation

Caution, Consult operation instructions

Humidity limitation

State of being fixed

atmospheric pressure limitation

## 15 Warranty

Our products are subjected to rigorous quality control and inspections. In the unlikely event of a product proving to be faulty within the guarantee period when it has been used normally, it will be repaired free of charge subject to the terms and conditions below. For such repair requests, please contact your Authorized NSK Dealer from whom the product was purchased.

Guarantee period.....One year

- In the following cases, the repair has to be paid for even if the product is within the guarantee period :
  - Fault and damage caused by misuse, or inappropriate repair or modification.
  - Fault and damage caused during transit or dropping after the product was purchased.
  - Fault and damage attributable to fire, earthquake, flooding, power surge, environmental pollution or other acts of providence.
  - Fault and damage caused by the use of consumables or methods other than those specified by NSK.
  - When the product has been repaired using components other than genuine components supplied by NSK.
- For repairs after the guarantee period, please contact your Authorized NSK Dealer from whom the product was purchased.

## 16 Optional Accessories

Product Name	Order Code	Quantity	Remarks
Cleaning Tube 6.0/L650	PD-CTTU-606501	1	-
Cleaning Tube 6.0/L50	PD-CTTU-600504	1	4 pcs per set.
Cleaning Tube 1.5/L25	PD-CTTU-150252	1	2 pcs per set.
Cleaning Holder	PD-CTH	1	Cleaning Tube 6.0/L650 and Cleaning Tube 6.0/L50 are included with this part.
Cleaning Adaptor Luer M	PD-CTAD-LM	1	-
Cleaning Adaptor SM	PD-CTAD-SM	1	-
Cleaning Adaptor 1T	PD-CTAD-1T	1	-
Cleaning Adaptor IN	PD-CTAD-IN	1	Cleaning Tube 1.5/L25 is included with this part.
Cleaning Adaptor G	PD-CTAD-G	1	-
PANA SPRAY Plus	Z182600	1	-

Product Name	Order Code	Quantity	Remarks
PANA SPRAY Plus (6pcs)	Y900630	1	6 pcs per set.
Spray Adaptor for Slim	P200-EZ-SM	1	For Standard Attachment 300. For Slim Attachment 300.
Irrigation Nozzle	PD-IN-D	1	For Standard Attachment 300.
	PD-IN-2ES	1	
	PD-IN-2S	1	
	PD-IN-2M	1	
	P200-IN-1A155	1	For Slim Attachment 300.
	P200-IN-1A175	1	
STD Attachment Beak	P300-2-B	1	For Standard Attachment 300.
Slim Tube Hood	P300-1T-H	1	For Slim Attachment 300.
Slim Tube Beak	P300-1T-B	1	

## 17 Disposing Product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

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