# Cleaning Tool for P300 Attachment

ENGLISH

## OPERATION MANUAL

Rx Only OM-SZ0907EN 000

This product includes cleaning tools that are designed for exclusive use with the P300 Attachment and Irrigation Nozzle. We recommend that you carefully read "12 Reprocessing" in the P300 Attachment Operation Manual prior to use to ensure proper product handling and use. Please keep this operation manual accessible for quick reference.

### 1 User and Indications for Use

- User** : Personnel with knowledge and skills required for operating this product.
- Indications for Use** : The Cleaning Tool is intended for use in rinsing, cleaning and purging lumens of the surgical attachment and the irrigation nozzles manufactured by Nakanishi INC.

### 2 Package Contents


One of the Cleaning Adaptors is included in the package.

Product Name	REF	Quantity
Cleaning Adaptor SM	PD-CTAD-SM	1
Cleaning Adaptor 1T	PD-CTAD-1T	1
Cleaning Adaptor IN*	PD-CTAD-IN	1
Operation Manual	-	1


\*Cleaning Tube 1.5/L25 is included with this part.

### 3 Cleaning System Figure

#### 3-1 Attachment / Slim Attachment Hub / Slim Tube Manual Cleaning System Figure

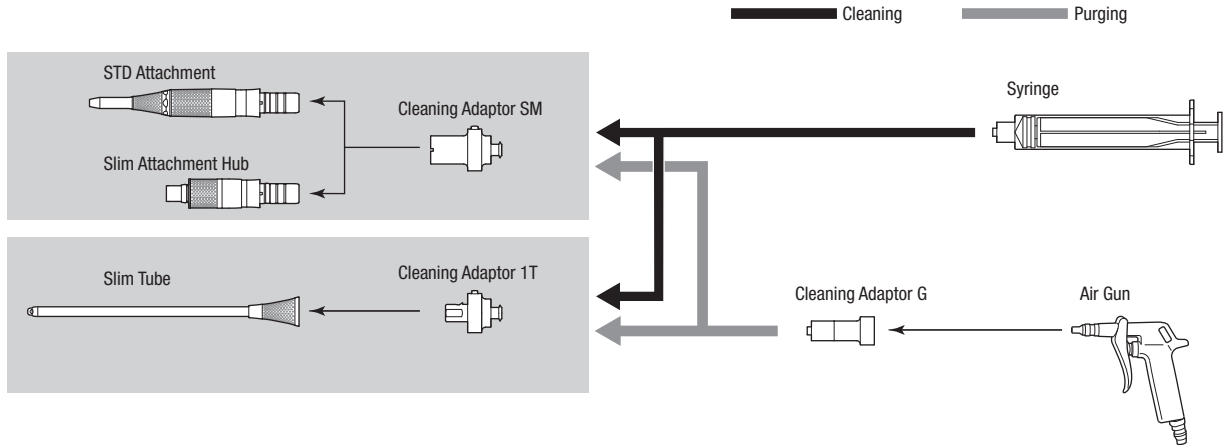
#### Automated Cleaning System Figure
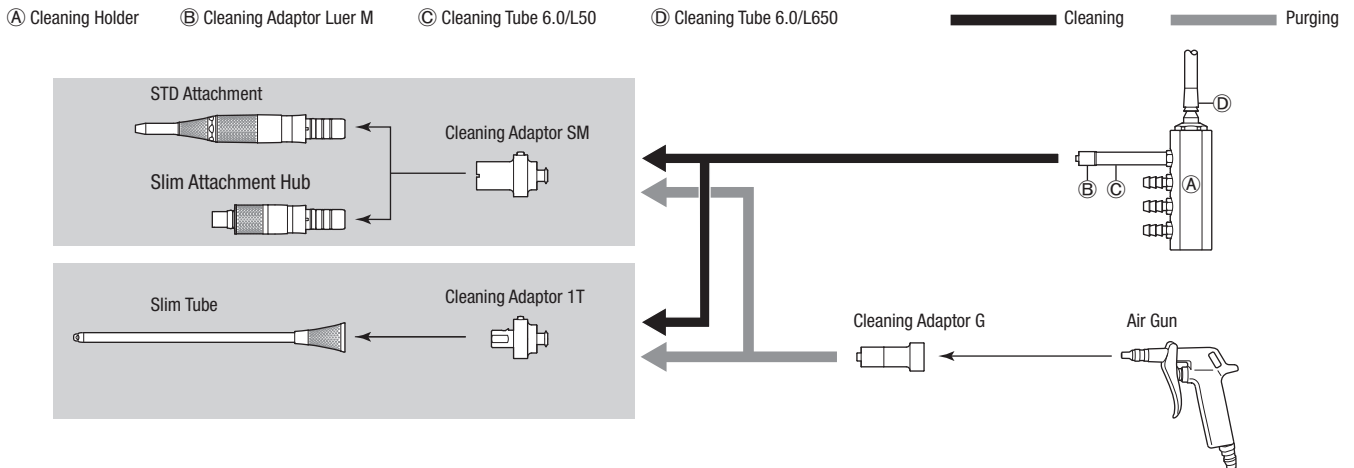
#### 3-2 Irrigation Nozzle

#### Manual Cleaning / Automated Cleaning System Figure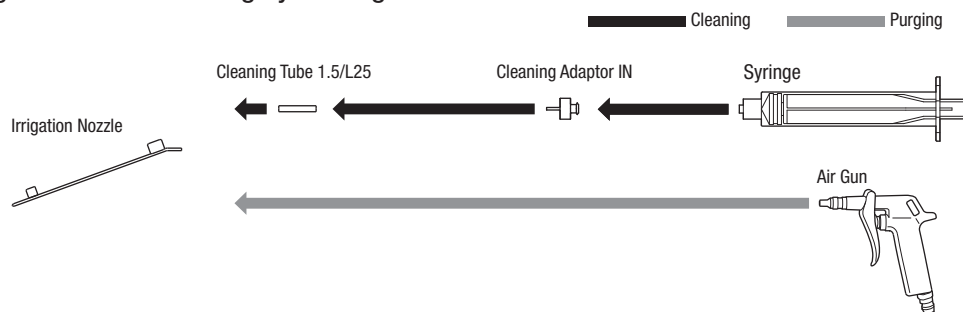

## 4 Operation

Refer to "12 Reprocessing" of the P300 Attachment Operation Manual (OM-SH0911EN) to ensure proper intended use.


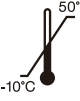
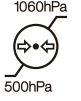
\* The validated effectiveness of reprocessing using these Cleaning Tools is only verified when used on P300 Attachments and Irrigation Nozzles.

Do not use the Cleaning Tools with other products.

## 5 Optional Accessories

Product Name	REF	Quantity	Remarks
Cleaning Tube 6.0/L650	PD-CTTU-606501	1	-
Cleaning Tube 6.0/L50	PD-CTTU-600504	1	4 pcs per set.
Cleaning Tube 1.5/L25	PD-CTTU-150252	1	2 pcs per set.
Cleaning Holder	PD-CTH	1	Cleaning Tube 6.0/L650 and Cleaning Tube 6.0/L50 are included with this part.
Cleaning Adaptor Luer M	PD-CTAD-LM	1	-
Cleaning Adaptor G	PD-CTAD-G	1	-

## 6 Use environment / Store and transportation environment

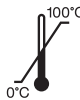Use environment			-
Store and transportation environment			

## 7 Symbols

Manufacturer

Date of manufacture

Mandatory conformance mark on many products placed on the market in the European Economic Area

Catalog number (Order Code)

Serial No.

Batch number

Medical device

Unique device identification

Authorized representative in the European community

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

See operation manual

Caution, consult operation instructions

GS1 DataMatrix for Unique Device Identifier

Temperature limitation

Humidity limitation

atmospheric pressure limitation

## 8 Disposing Product

In order to avoid health risks to the operators and environmental contamination caused by the handling and disposing of medical equipment, a healthcare professional is required to confirm the equipment is sterile. Disposal of medical equipment, especially controlled medical wastes, should be handled by a licensed professional.

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#### 3-1 Standard Attachment 300, Slim Attachment 300

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