

DI129A-EN (Rev K)	BEACON® STABILIZATION SYSTEM
05/2025	IMPORTANT INFORMATION ON THE BEACON® STABILIZATION SYSTEM
 GLOBUS M E D I C A L  GLOBUS MEDICAL, INC. Valley Forge Business Center 2560 General Armistead Avenue Audubon, PA 19403 USA Customer Service: Phone 1-866-GLOBUS1 (OR) 1-866-456-2871 Fax 1-866-GLOBUS3 (OR) 1-866-456-2873	

For symbols glossary, please refer to www.globusmedical.com/eIFU

ENGLISH

WITHIN THE UNITED STATES ONLY

IMPORTANT INFORMATION ON THE BEACON® STABILIZATION SYSTEM

DESCRIPTION

The BEACON® Stabilization System consists of rods, posted screws, reduction screws, clamps, ML connectors, other connectors, and associated manual surgical instruments. Screws and rods are available in a variety of sizes to accommodate individual patient anatomy. Implant components can be rigidly locked into a variety of configurations for the individual patient and surgical condition. Screws, clamps, and rods may be used anteriorly or posteriorly. Connectors are intended for posterior use only. Preassembled clamps are used to connect screws to the rod.

The most common use of this screw and rod system in the posterior thoracolumbar and sacral spine is two rods, each positioned and attached lateral to the spinous process via pedicle screws. The most common use of this screw and rod system in the anterior thoracolumbar spine is one rod, positioned and attached to the vertebral bodies via screws through an appropriate size staple.

The rods are composed of titanium alloy, commercially pure titanium or cobalt chromium-molybdenum alloy as specified in ASTM F136, F1295, F67, and F1537. All other BEACON® implants are composed of titanium alloy, as specified in ASTM F136 and F1295. The screws are available with or without hydroxyapatite (HA) coating, as specified in ASTM F1185. Due to the risk of galvanic corrosion following implantation, stainless steel implants should not be connected to titanium, titanium alloy or cobalt chromium-molybdenum alloy implants.

INDICATIONS

The BEACON® Stabilization System, when used as posterior pedicle screw systems, is intended to provide immobilization and stabilization of spinal segments in skeletally mature patients as an adjunct to fusion in the treatment of the following acute and chronic instabilities or deformities of the thoracic, lumbar and sacral spine: degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies), degenerative spondylolisthesis with objective evidence of neurologic impairment, fracture, dislocation, scoliosis, kyphosis, spinal tumor, pseudoarthrosis and failed previous fusion.

In addition, the BEACON® Stabilization System is intended for the treatment of severe spondylolisthesis (Grades 3 and 4) of the L5-S1 vertebra in skeletally mature patients receiving fusion by autogenous bone graft, having implants attached to the lumbosacral spine and/or ilium with removal of the implants after attainment of a solid fusion. Levels of pedicle screw fixation for these patients are L3-sacrum/iliacum.

When used as a posterior non-pedicle screw fixation system (using REVERE® hooks), the BEACON® Stabilization System is intended for the treatment of degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies), spinal stenosis, spondylolisthesis, spinal deformities (i.e. scoliosis, kyphosis, and/or lordosis, Scheuermann's disease), fracture, pseudoarthrosis, tumor resection, and/or failed previous fusion. Overall levels of fixation are T1-sacrum/iliacum.

When used as an anterolateral thoracolumbar system, the BEACON® Stabilization System is intended for anterolateral screw (with or without staple) fixation for the following indications: degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies), spinal stenosis, spondylolisthesis, spinal deformities (i.e. scoliosis, kyphosis, and/or lordosis), fracture or dislocation of the thoracolumbar spine, pseudoarthrosis, tumor resection, and/or failed previous fusion. Levels of screw fixation are T8-L5.

CONTRAINDICATIONS

Certain degenerative diseases or underlying physiological conditions such as diabetes or rheumatoid arthritis may alter the healing process, thereby increasing the risk of implant breakage.

Mental or physical impairment which compromises a patient's ability to comply with necessary limitations or precautions may place that patient at a particular risk during postoperative rehabilitation.

Factors such as the patient's weight, activity level, and adherence to weight bearing or load bearing instructions have an effect on the stresses to which the implant is subjected.

WARNINGS

The safety and effectiveness of pedicle screw spinal systems have been established only for spinal conditions with significant mechanical instability or deformity requiring fusion with instrumentation. These conditions are significant mechanical instability or deformity of the thoracic, lumbar, and sacral spine secondary to degenerative disc disease, degenerative spondylolisthesis with objective evidence of neurologic impairment, fracture, dislocation, scoliosis, kyphosis, spinal tumor and failed previous fusion (pseudoarthrosis). The safety and effectiveness of these devices for any other conditions are unknown.

Possible adverse effects which may occur include: failed fusion or pseudoarthrosis leading to implant breakage; allergic reaction to implant materials; device fracture or failure; device migration or loosening; decrease in bone density; vertebral fracture, pain, discomfort, or abnormal sensations due to the presence of the device; injury to nerves, vessels, and organs; venous thrombosis, lung embolism and cardiac arrest; and death.

The components of this system are manufactured from titanium alloy, commercially pure titanium, and cobalt chromiummolybdenum alloy. Mixing of stainless steel implant components with different materials is not recommended for metallurgical, mechanical and functional reasons.

These warnings do not include all adverse effects which could occur with surgery in general, but are important considerations particular to orthopedic implants. General surgical risks should be explained to the patient prior to surgery.

PRECAUTIONS

Implantation of screw and plate systems should be performed only by experienced spinal surgeons with specific training in the use of this system because this is a technically demanding procedure presenting a risk of serious injury to the patient. Preoperative planning and patient anatomy should be considered when selecting screw diameter and length.

The BEACON® System includes a 5.5mm rod and a 6.35mm rod. All implants in this system are intended for use with either a 5.5mm rod or a 6.35mm rod.

Surgical implants must never be reused. An explanted metal implant must never be reimplanted. Even though the device appears undamaged, it may have small defects and internal stress patterns which could lead to breakage.

Correct handling of the implant is extremely important. Contouring of metal implants should be avoided where possible. If contouring is necessary, or allowed by design, the surgeon should avoid sharp bends, reverse bends, or bending the device at a screw hole. The operating surgeon should avoid any notching or scratching of the device when contouring it. These factors may produce internal stresses which may become the focal point for eventual breakage of the implant.

Metallic implants can loosen, fracture, corrode, migrate, cause pain, or stress shield bone even after a fracture has healed, particularly in young, active patients. While the surgeon must have the final decision on implant removal, we recommend that whenever possible and practical for the individual patient, fixation devices should be removed once their service as an aid to healing is accomplished. Implant removal should be followed by adequate postoperative management.

The BEACON® Stabilization System has not been evaluated for safety and compatibility in the MR environment. The BEACON® Stabilization System has not been tested for heating or migration in the MR environment.

PACKAGING

These implants and instruments may be supplied pre-packaged and sterile, using gamma irradiation. The integrity of the sterile packaging should be checked to ensure that sterility of the contents is not compromised. Packaging should be carefully checked for completeness and all components should be carefully checked to ensure that there is no damage prior to use. Damaged packages or products should not be used, and should be returned to Globus Medical. During surgery, after the correct size has been determined, remove the products from the packaging using aseptic technique.

The instrument sets are provided nonsterile and are steam sterilized prior to use, as described in the STERILIZATION section below. Following use or exposure to soil, instruments must be cleaned, as described in the CLEANING section below.

HANDLING

All instruments and implants should be treated with care. Improper use or handling may lead to damage and/or possible malfunction. Products should be checked to ensure that they are in working order prior to surgery. All products should be inspected prior to use to ensure that there is no unacceptable deterioration such as corrosion, discoloration, pitting, cracked seals, etc. Non-working or damaged instruments should not be used, and should be returned to Globus Medical.

CLEANING

All instruments that can be disassembled must be disassembled for cleaning. All handles must be detached. Instruments may be reassembled following sterilization. The instruments should be cleaned using neutral cleaners before sterilization and introduction into a sterile surgical field or (if applicable) return of the product to Globus Medical.

Cleaning and disinfecting of instruments can be performed with aldehyde-free solvents at higher temperatures. Cleaning and decontamination must include the use of neutral cleaners followed by a deionized water rinse. Note: certain cleaning solutions such as those containing formalin, glutaraldehyde, bleach and/or other alkaline cleaners may damage some devices, particularly instruments; these solutions should not be used.

The following cleaning methods should be observed when cleaning instruments after use or exposure to soil, and prior to sterilization:

1. Immediately following use, ensure that the instruments are wiped down to remove all visible soil and kept from drying by submerging or covering with a wet towel.
2. Disassemble all instruments that can be disassembled.
3. Rinse the instruments under running tap water to remove all visible soil. Flush the lumens a minimum of 3 times, until the lumens flush clean.
4. Prepare Enzol® (or a similar enzymatic detergent) per manufacturer's recommendations.
5. Immerse the instruments in the detergent and allow them to soak for a minimum of 2 minutes.
6. Use a soft bristled brush to thoroughly clean the instruments. Use a pipe cleaner for any lumens. Pay close attention to hard to reach areas.
7. Using a sterile syringe, draw up the enzymatic detergent solution. Flush any lumens and hard to reach areas until no soil is seen exiting the area.
8. Remove the instruments from the detergent and rinse them in running warm tap water.
9. Prepare Enzol® (or a similar enzymatic detergent) per manufacturer's recommendations in an ultrasonic cleaner.
10. Completely immerse the instruments in the ultrasonic cleaner and ensure detergent is in lumens by flushing the lumens. Sonicate for a minimum of 3 minutes.
11. Remove the instruments from the detergent and rinse them in running deionized water or reverse osmosis water for a minimum of 2 minutes.
12. Dry instruments using a clean soft cloth and filtered pressurized air.
13. Visually inspect each instrument for visible soil. If visible soil is present, then repeat cleaning process starting with Step 3.

CONTACT INFORMATION

Globus Medical may be contacted at 1-866-GLOBUS1 (456-2871). A surgical technique manual may be obtained by contacting Globus Medical.

STERILIZATION

These implants and instruments may be available sterile or nonsterile. HA-coated implants are only available sterile.

Sterile implants and instruments are sterilized by gamma radiation, validated to ensure a Sterility Assurance Level (SAL) of 10⁻⁶. Sterile products are packaged in a heat sealed, double foil pouch. The expiration date is provided in the package label. These products are considered sterile unless the packaging has been opened or damaged.

Nonsterile implants and instruments have been validated to ensure an SAL of 10⁻⁶. The use of an FDA-cleared wrap is recommended, per the Association for the Advancement of Medical Instrumentation (AAMI) ST79, *Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities*. It is the end user's responsibility to use only sterilizers and accessories (such as sterilization wraps, sterilization pouches, chemical indicators, biological indicators, and sterilization cassettes) that have been cleared by the FDA for the selected sterilization cycle specifications (time and temperature).

When using a rigid sterilization container, the following must be taken into consideration for proper sterilization of Globus devices and loaded graphic cases:

- Recommended sterilization parameters are listed in the table below.
- Only FDA-cleared rigid sterilization containers for use with pre-vacuum steam sterilization may be used.
- When selecting a rigid sterilization container, it must have a minimum filter area of 176 in² total, or a minimum of four (4) 7.5in diameter filters.
- No more than one (1) loaded graphic case or its contents can be placed directly into a rigid sterilization container.
- Stand-alone modules/racks or single devices must be placed, without stacking, in a container basket to ensure optimal ventilation.
- The rigid sterilization container manufacturer's instructions for use are to be followed; if questions arise, contact the manufacturer of the specific container for guidance.
- Refer to AAMI ST79 for additional information concerning the use of rigid sterilization containers..

For implants and instruments provided NONSTERILE, sterilization is recommended (wrapped or containerized) as follows:

Method	Cycle Type	Temperature	Exposure Time	Drying Time
Steam	Pre-vacuum	132°C (270°F)	4 minutes	30 minutes

These parameters are validated to sterilize only this device. If other products are added to the sterilizer, the recommended parameters are not valid and new cycle parameters must be established by the user. The sterilizer must be properly installed, maintained, and calibrated. Ongoing testing must be performed to confirm inactivation of all forms of viable microorganisms.

CAUTION: Federal (U.S.A.) Law Restricts this Device to Sale by or on the Order of a Physician.