

DI121B-EN (Rev R)	PATRIOT™ LUMBAR SPACERS
05/2025	IMPORTANT INFORMATION ON PATRIOT™ LUMBAR SPACERS
 GLOBUS M E D I C A L  GLOBUS MEDICAL, INC. Valley Forge Business Center 2560 General Armistead Avenue Audubon, PA 19403 USA Customer Service: Phone 1-866-GLOBUS1 (OR) 1-866-456-2871 Fax 1-866-GLOBUS3 (OR) 1-866-456-2873	EC REP: AJW Technology Consulting GmbH Breite Straße 3 40213 Düsseldorf, Germany CH REP: AJW Technology Consulting GmbH Kreuzplatz 2, 8032 Zurich, Switzerland AUSTRALIA SPONSOR: GLOBUS MEDICAL AUSTRALIA PTY LIMITED, Unit 9/5-7 Inglewood Place Baulkham Hills NSW 2153, Australia   0297 

For symbols glossary, please refer to www.globusmedical.com/eIFU

ENGLISH

OUTSIDE THE UNITED STATES ONLY

IMPORTANT INFORMATION ON PATRIOT™ LUMBAR SPACERS

DESCRIPTION

The PATRIOT™ Spacers (CONSTITUTION™ PLIF, SIGNATURE™ TLIF, CONTINENTAL™ ALIF, TransContinental™ M and TransContinental™ Spacers) are lumbar interbody fusion devices used to provide structural stability in skeletally mature individuals following discectomy. Each of the PATRIOT™ Spacers provides a different shape to accommodate various surgical approaches to the lumbar spine (posterior, transforaminal, anterior, and lateral). The devices are available in various heights and geometric options to fit the anatomical needs of a wide variety of patients. These spacers are to be filled with autogenous bone graft material. Protrusions on the superior and inferior surfaces of each device grip the endplates of the adjacent vertebrae to resist expulsion.

PATRIOT™ Spacers are made from commercially pure titanium, titanium alloy, or radiolucent polymer, as specified in ASTM F2026, F136, F1295 and F67. Each PEEK spacer contains titanium alloy or tantalum markers, as specified in ASTM F560. All PEEK implants are additionally available with a commercially pure titanium plasma spray coating, as specified in ASTM F67 and F1580.

INDICATIONS

PATRIOT™ Spacers (CONSTITUTION™ PLIF, SIGNATURE™ TLIF, CONTINENTAL™ ALIF, TransContinental™ and TransContinental™ M Spacers) are interbody fusion devices intended for use in skeletally mature patients with degenerative disc disease (DDD) at one or more levels of the lumbosacral spine (L2-S1). DDD is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. In addition, these patients may have up to Grade 1 spondylolisthesis or retrolisthesis at the involved level(s). PATRIOT™ Spacers are to be filled with bone graft material. These devices are intended to be used with supplemental fixation, such as the PROTEX™ or REVERE™ Stabilization System.

WARNINGS

One of the potential risks identified with this system is death. Other potential risks which may require additional surgery, include:

- device component fracture,
- loss of fixation,
- non-union,
- fracture of the vertebrae,
- neurological injury, and
- vascular or visceral injury.

Certain degenerative diseases or underlying physiological conditions such as diabetes, rheumatoid arthritis, or osteoporosis may alter the healing process, thereby increasing the risk of implant breakage or spinal fracture.

Patients with previous spinal surgery at the level(s) to be treated may have different clinical outcomes compared to those without previous surgery.

Components of this system should not be used with components of any other system or manufacturer.

The components of this system are manufactured from PEEK radiolucent polymer, titanium, titanium alloy and tantalum. Dissimilar metals in contact with each other can accelerate the corrosion process due to galvanic corrosion effects. Mixing of implant components with different materials is not recommended for metallurgical, mechanical and functional reasons.

These warnings do not include all adverse effects which could occur with surgery in general, but are important considerations particular to orthopedic implants. General surgical risks should be explained to the patient prior to surgery.

PRECAUTIONS

The implantation of intervertebral fusion devices should be performed only by experienced spinal surgeons because this is a technically demanding procedure presenting a risk of serious injury to the patient. Preoperative planning and patient anatomy should be considered when selecting implant size.

Surgical implants must never be reused. An explanted implant must never be reimplanted. Even though the device appears undamaged, it may have small defects and internal stress patterns which could lead to breakage.

Adequately instruct the patient. Mental or physical impairment which compromises or prevents a patient's ability to comply with necessary limitations or precautions may place that patient at a particular risk during postoperative rehabilitation.

For optimal implant performance, the surgeon should consider the levels of implantation, patient weight, patient activity level, other patient conditions, etc., which may impact on the performance of this system.

MRI SAFETY INFORMATION



The PATRIOT™ Spacers (CONSTITUTION™ PLIF, SIGNATURE™ TLIF, CONTINENTAL™ ALIF, TransContinental™ M and TransContinental™ Spacers) are MR Conditional. A patient with these devices can be safely scanned in an MR system meeting the following conditions:

- Static magnetic field of 1.5 Tesla and 3.0 Tesla only
- Maximum spatial field gradient of 3,000 gauss/cm (30 T/m) or less
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 1 W/kg

Under the scan conditions defined above, the PATRIOT™ Spacers (CONSTITUTION™ PLIF, SIGNATURE™ TLIF, CONTINENTAL™ ALIF, TransContinental™ M and TransContinental™ Spacers) are expected to produce a maximum temperature rise of less than or equal to 3.9°C after 15 minutes of continuous scanning.

The image artifact caused by these devices is not expected to extend beyond 35mm from the device when imaged with a gradient echo pulse sequence and a 3.0 Tesla MRI system.

CONTRAINDICATIONS

Use of these devices is contraindicated in patients with the following conditions:

1. Active systemic infection, infection localized to the site of the proposed implantation, or when the patient has demonstrated allergy or foreign body sensitivity to any of the implant materials.
2. Prior fusion at the level(s) to be treated.
3. Severe osteoporosis, which may prevent adequate fixation
4. Conditions that may place excessive stresses on bone and implants, such as severe obesity or degenerative diseases, are relative contraindications. The decision whether to use these devices in such conditions must be made by the physician taking into account the risks versus the benefits to the patient.
5. Patients whose activity, mental capacity, mental illness, alcoholism, drug abuse, occupation, or lifestyle that may interfere with their ability to follow postoperative restrictions and who may place undue stresses on the implant during bony healing may be at a higher risk of implant failure.
6. Any patient not willing to cooperate with postoperative instructions.
7. Any condition not described in the indications for use.
8. Fever or leukocytosis.
9. Pregnancy.
10. Any other condition that would preclude the potential benefit of spinal implant surgery, such as the presence of tumors or congenital abnormalities, fracture local to the operating site, elevation of sedimentation rate unexplained by other diseases, elevations of the white blood count (WBC), or a marked left shift in the WBC differential count.
11. Any case not needing a fusion.
12. Patients with a known hereditary or acquired bone friability or calcification problem should not be considered for this type of surgery.
13. These devices must not be used for pediatric cases or where the patient still has general skeletal growth.
14. Spondylolisthesis unable to be reduced to Grade I.
15. Any case where the implant components selected for used would be too large or too small to achieve a successful result.
16. Any case that requires the mixing of metals from two different components or systems.
17. Any patient having inadequate tissue coverage at the operative site or inadequate bone stock or quality.
18. Any patient in which implant utilization would interfere with anatomical structures or expected physiological performance.

COMPLICATIONS AND POSSIBLE ADVERSE EVENTS

Prior to surgery, patients should be made aware of the following possible adverse effects in addition to the potential need for additional surgery to correct these effects:

- Loosening, bending or breakage of components
- Displacement/migration of device components
- Tissue sensitivity to implant material
- Potential for skin breakdown and/or wound complications
- Non-union or delayed union or mal-union
- Infection
- Nerve damage, including loss of neurological function (sensory and/or motor), paralysis, dysesthesia, hyperesthesia, paresthesia, radiculopathy, reflex deficit, cauda equina syndrome
- Dural tears, cerebral spinal fluid leakage
- Fracture of vertebrae

- Foreign body reaction (allergic) to components or debris
- Vascular or visceral injury
- Change in spinal curvature, loss of correction, height and/or reduction
- Urinary retention or loss of bladder control or other types of disorders of the urogenital system
- Ileus, gastritis, bowel obstruction or other types of gastrointestinal system compromise
- Reproductive system compromise including impotence, sterility, loss of consortium and sexual dysfunction.
- Pain or discomfort
- Bursitis
- Decrease in bone density due to stress shielding
- Loss of bone or fracture of bone above or below the level of surgery
- Bone graft donor site pain, fracture, and/or delayed wound healing
- Restriction of activities
- Lack of effective treatment of symptoms for which surgery was intended
- Need for additional surgical intervention
- Death

PACKAGING

These implants and instruments may be supplied pre-packaged and sterile, using gamma irradiation. The integrity of the sterile packaging should be checked to ensure that sterility of the contents is not compromised. Packaging should be carefully checked for completeness and all components should be carefully checked to ensure that there is no damage prior to use. Damaged packages or products should not be used, and should be returned to Globus Medical. During surgery, after the correct size has been determined, remove the products from the packaging using aseptic technique.

The instrument sets are provided nonsterile and are steam sterilized prior to use, as described in the STERILIZATION section below. Following use or exposure to soil, instruments must be cleaned, as described in the CLEANING section below.

HANDLING

All instruments and implants should be treated with care. Improper use or handling may lead to damage and/or possible malfunction. Products should be checked to ensure that they are in working order prior to surgery. All products should be inspected prior to use to ensure that there is no unacceptable deterioration such as corrosion, discoloration, pitting, cracked seals, etc. Non-working or damaged instruments should not be used, and should be returned to Globus Medical.

CLEANING

All instruments that can be disassembled must be disassembled for cleaning. All handles must be detached. Instruments may be reassembled following sterilization. The instruments should be cleaned using neutral cleaners before sterilization and introduction into a sterile surgical field or (if applicable) return of the product to Globus Medical.

Cleaning and disinfecting of instruments can be performed with aldehyde-free solvents at higher temperatures. Cleaning and decontamination must include the use of neutral cleaners followed by a deionized water rinse. Note: certain cleaning solutions such as those containing formalin, glutaraldehyde, bleach and/or other alkaline cleaners may damage some devices, particularly instruments; these solutions should not be used.

The following cleaning methods should be observed when cleaning instruments after use or exposure to soil, and prior to sterilization:

1. Immediately following use, ensure that the instruments are wiped down to remove all visible soil and kept from drying by submerging or covering with a wet towel.
2. Disassemble all instruments that can be disassembled.
3. Rinse the instruments under running tap water to remove all visible soil. Flush the lumens a minimum of 3 times, until the lumens flush clean.
4. Prepare Enzo[®] (or a similar enzymatic detergent) per manufacturer's recommendations.
5. Immerse the instruments in the detergent and allow them to soak for a minimum of 2 minutes.
6. Use a soft bristled brush to thoroughly clean the instruments. Use a pipe cleaner for any lumens. Pay close attention to hard to reach areas.
7. Using a sterile syringe, draw up the enzymatic detergent solution. Flush any lumens and hard to reach areas until no soil is seen exiting the area.
8. Remove the instruments from the detergent and rinse them in running warm tap water.
9. Prepare Enzo[®] (or a similar enzymatic detergent) per manufacturer's recommendations in an ultrasonic cleaner.
10. Completely immerse the instruments in the ultrasonic cleaner and ensure detergent is in lumens by flushing the lumens. Sonicate for a minimum of 3 minutes.
11. Remove the instruments from the detergent and rinse them in running deionized water or reverse osmosis water for a minimum of 2 minutes.
12. Dry instruments using a clean soft cloth and filtered pressurized air.
13. Visually inspect each instrument for visible soil. If visible soil is present, then repeat cleaning process starting with Step 3.

CONTACT INFORMATION

Globus Medical may be contacted at 1-866-GLOBUS1 (456-2871). A surgical technique manual may be obtained by contacting Globus Medical.

STERILIZATION

These implants and instruments may be available sterile or nonsterile, with the exception of coated implants, which are only available sterile.

Sterile implants and instruments are sterilized by gamma radiation, validated to ensure a Sterility Assurance Level (SAL) of 10⁻⁶. Sterile products are packaged in a heat sealed,

double foil pouch. The expiration date is provided in the package label. These products are considered sterile unless the packaging has been opened or damaged.

Nonsterile implants and instruments have been validated to ensure an SAL of 10⁻⁶. The use of a wrap is recommended, per the Association for the Advancement of Medical Instrumentation (AAMI) ST79, *Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities*. It is the end user's responsibility to use only sterilizers and accessories (such as sterilization wraps, sterilization pouches, chemical indicators, biological indicators, and sterilization cassettes) that are designed for the selected sterilization cycle specifications (time and temperature).

When using a rigid sterilization container, the following must be taken into consideration for proper sterilization of Globus devices and loaded graphic cases:

- Recommended sterilization parameters are listed in the table below.
- Only rigid sterilization containers for use with pre-vacuum steam sterilization may be used.
- When selecting a rigid sterilization container, it must have a minimum filter area of 176 in² total, or a minimum of four (4) 7.5in diameter filters.
- No more than one (1) loaded graphic case or its contents can be placed directly into a rigid sterilization container.
- Stand-alone modules/racks or single devices must be placed, without stacking, in a container basket to ensure optimal ventilation.
- The rigid sterilization container manufacturer's instructions for use are to be followed; if questions arise, contact the manufacturer of the specific container for guidance.
- Refer to AAMI ST79 for additional information concerning the use of rigid sterilization containers.

For implants and instruments provided NONSTERILE, sterilization is recommended (wrapped or containerized) as follows:

Method	Cycle Type	Temperature	Exposure Time	Drying Time
Steam	Pre-vacuum	132°C (270°F)	4 minutes	30 minutes
Steam	Pre-vacuum	134°C (273°F)	3 minutes	30 minutes

These parameters are validated to sterilize only this device. If other products are added to the sterilizer, the recommended parameters are not valid and new cycle parameters must be established by the user. The sterilizer must be properly installed, maintained, and calibrated. Ongoing testing must be performed to confirm inactivation of all forms of viable microorganisms.