

| DI247A-EN<br>(Rev A)                                                                                                                                                                                                                                                                                                                                                                                                                                                                                 | TENCEL™ SACROILIAC JOINT FIXATION<br>SYSTEM                                      |  |
|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------|--|
| 07/2026<br><br>Distributed by<br><br><br><br>1800 Bridge Pkwy, Redwood City,<br>CA 94065 USA<br><br> GLOBUS MEDICAL, INC.<br>Valley Forge Business Center<br>2560 General Armistead Avenue<br>Audubon, PA 19403<br>USA<br>Customer Service:<br>Phone 1-866-GLOBUS1 (OR)<br>1-866-456-2871<br>Fax 1-866-GLOBUS3 (OR)<br>1-866-456-2873 | <b>IMPORTANT INFORMATION ON THE TENCEL™<br/>SACROILIAC JOINT FIXATION SYSTEM</b> |  |
|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |                                                                                  |  |

For symbols glossary, please refer to [www.globusmedical.com/eIFU](http://www.globusmedical.com/eIFU)

## ENGLISH

### WITHIN THE UNITED STATES ONLY

### IMPORTANT INFORMATION ON THE TENCEL™ SACROILIAC JOINT FIXATION SYSTEM

#### DESCRIPTION

The TENCEL™ Sacroiliac Joint Fixation System (including TENCEL™ Select) consists of screws designed to enhance sacroiliac joint fusion and to provide fixation of large bones and large bone fragments of the pelvis. TENCEL™ screws are cannulated, partially threaded or fully threaded screws that contain a pre-assembled contouring washer. TENCEL™ Select screws are cannulated, fully threaded screws that are offered with or without a pre-assembled contouring washer, in various diameters and lengths to accommodate patient anatomy. A minimum of two screws are required for the lateral approach to achieve stabilization. A third screw may be used as needed for additional fixation. One to three screws may be used for the posterior approaches.

The TENCEL™ Sacroiliac Joint Fixation System screws and contouring washers are manufactured from titanium alloy, as specified in ASTM F136 and F1295. TENCEL™ screws are available with or without hydroxyapatite (HA) coated, as specified in ASTM F1185.

The TENCEL™ Sacroiliac Joint Fixation System includes manual surgical instruments manufactured from stainless steel, as specified in ASTM F899.

#### INDICATIONS

The TENCEL™ Sacroiliac Joint Fixation System is intended for sacroiliac joint fusion for conditions including sacroiliac joint disruptions and degenerative sacroiliitis.

#### WARNINGS

One of the potential risks identified with this system is death. Other potential risks which may require additional surgery include:

Possible adverse effects which may occur include, but are not limited to: failed fusion or pseudarthrosis leading to implant breakage; allergic reaction to implant materials including metallosis, staining, tumor formation and/or autoimmune disease; infection; device fracture or failure; device migration or loosening; decrease in bone density; loss of spinal mobility or function; inability to perform activities of daily living; graft donor site complications including pain, fracture and wound healing problems; tissue damage, pain, discomfort, or abnormal sensations due to the presence of the device or implantation surgery; scar formation causing neurologic compromise or pain; injury to nerves including loss or decrease of neurologic function, paralysis, numbness or tingling; cauda equina syndrome; injury to vessels, hemorrhage, hematoma, occlusion, seroma, edema, embolism, stroke, or other types of cardiovascular system compromise; injury to organs including urinary retention, loss of bladder control, or other types of urologic system compromise; gastrointestinal system compromise; reproductive system compromise including sterility, sexual dysfunction; development of respiratory problems including pulmonary embolism; venous thrombosis, lung embolism and cardiac arrest; and death. Additional surgery may be necessary to correct some of these effects. Women of childbearing potential should be cautioned that vaginal delivery of a fetus may not be advisable following SI joint fusion. If pregnancy occurs, delivery options should be discussed with her obstetrician. Certain degenerative diseases or underlying physiological conditions such as diabetes or rheumatoid arthritis may alter the healing process, thereby increasing the risk of fracture fixation of large bones and large bone fragments of the pelvis.

These warnings do not include all adverse effects which could occur with surgery in general, but are important considerations particular to orthopedic implants. General surgical risks should be explained to the patient prior to surgery.

The components of this system are manufactured from titanium alloy. Mixing of implant components with different materials is not recommended, for metallurgical, mechanical and functional reasons.

#### PRECAUTIONS

Use of the TENCEL™ Sacroiliac Joint Fixation System should be performed only by experienced healthcare professionals with specific training in the use of this system due to a risk of serious injury to the patient. Preoperative planning and patient anatomy should be considered prior to performing the sacroiliac joint fusion.

Adequately instruct the patient. Mental or physical impairment which compromises or prevents a patient's ability to comply with necessary limitations or precautions may place that patient at a particular risk during postoperative rehabilitation.

Surgical implants are SINGLE USE ONLY and must never be reused. An explanted implant must never be reimplanted. Even though the device appears undamaged, it may have small defects and internal stress patterns which could lead to breakage.

For optimal implant performance, when using the TENCEL™ Sacroiliac Joint Fixation System, the physician/surgeon should consider the levels of implantation, patient weight, patient activity level, other patient conditions, etc., which may impact on the performance of this system.

Extreme caution should be used around the nerve roots. Damage to the nerves will cause loss of neurological functions. Whenever possible or necessary, an imaging system should be utilized to facilitate surgery. To insert a cannulated screw, a guide wire may be used, followed by a sharp tap. Ensure that the guide wire, if used, is not inserted too deep, becomes bent, and/or breaks. Also ensure that the guide wire does not advance during tapping or screw insertion. Remove the guide wire and confirm that it is intact. Failure to do so may cause the guide wire or part of it to advance through the bone and into a location that may cause damage to underlying structures.

Correct handling of the implant is extremely important. Metallic implants can loosen, fracture, corrode, migrate, cause pain, or stress shield bone even after a fracture or has healed or fusion has occurred, particularly in young, active patients. While the healthcare professional must have the final decision on implant removal, we recommend that whenever possible and practical for the individual patient, fixation devices should be removed once their service as an aid to healing is accomplished. Implant removal should be followed by adequate postoperative management.

Metallic internal fixation devices cannot withstand the activity levels and/or loads equal to those placed on normal, healthy bone. A higher risk of device loosening, bending, or breaking exists with fractures involving severe comminution, displacement or other difficult fracture management situations.

Corrosion of the implant can occur. Implanting metals and alloys in the human body subjects them to constantly changing environment of salts, acids and alkalis, which can cause corrosion. Placing dissimilar metals in contact with each other can accelerate the corrosion process, which in turn can enhance fatigue fractures of implants. Thus every effort should be made to use compatible metals and alloys in conjunction with each other.

#### MRI SAFETY INFORMATION

These devices have not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of devices in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

#### CONTRAINDICATIONS

The contraindications include, but are not limited to: active infectious process or significant risk of infection (immunocompromise); local inflammation, fever, or leukocytosis, morbid obesity; pregnancy; mental illness; distorted anatomy caused by congenital abnormalities; any medical or surgical condition which would preclude the potential benefit of spinal implant surgery, such as the presence of tumors or congenital abnormalities; rapid joint disease, bone absorption osteopenia, and/or osteoporosis; suspected or documented metal allergy or intolerance; any case where metals must be mixed from different components; any case where the implant components selected for use would be too large or too small to achieve a successful result; any case where fracture healing is not required; any patient in which implant utilization would interfere with anatomical structures or expected physiological performance; any patient unwilling to follow post-operative instructions; any case not described in the indications.

Certain degenerative diseases or underlying physiological conditions such as diabetes or rheumatoid arthritis may alter the healing process, thereby increasing the risk of implant breakage.

Mental or physical impairment which compromises a patient's ability to comply with necessary limitations or precautions may place that patient at a particular risk during postoperative rehabilitation.

Factors such as the patient's weight, activity level, and adherence to weight bearing or load bearing instructions have an effect on the stresses to which the implant is subjected.

#### PACKAGING

These implants may be supplied pre-packaged and sterile, using gamma irradiation. The integrity of the sterile packaging should be checked to ensure that sterility of the contents is not compromised. Packaging should be carefully checked for completeness and all components should be carefully checked to ensure that there is no damage prior to use. Damaged packages or products should not be used, and should be returned to Globus Medical. During surgery, after the correct size has been determined, remove the products from the packaging using aseptic technique.

The instrument sets are provided nonsterile and are steam sterilized prior to use, as described in the STERILIZATION section below. Following use or exposure to soil, instruments must be cleaned, as described in the CLEANING section below.

## HANDLING

All instruments and implants should be treated with care. Improper use or handling may lead to damage and/or possible malfunction. Products should be checked to ensure that they are in working order prior to surgery. All products should be inspected prior to use to ensure that there is no unacceptable deterioration such as corrosion, discoloration, pitting, cracked seals, etc. Non-working or damaged instruments should not be used, and should be returned to Globus Medical.

## CLEANING

All instruments that can be disassembled must be disassembled for cleaning. All handles must be detached. Instruments may be reassembled following sterilization. The instruments should be cleaned using neutral cleaners before sterilization and introduction into a sterile surgical field or (if applicable) return of the product to Globus Medical.

Cleaning and disinfecting of instruments can be performed with aldehyde-free solvents at higher temperatures. Cleaning and decontamination must include the use of neutral cleaners followed by a deionized water rinse. Note: certain cleaning solutions such as those containing formalin, glutaraldehyde, bleach and/or other alkaline cleaners may damage some devices, particularly instruments; these solutions should not be used.

The following cleaning methods should be observed when cleaning instruments after use or exposure to soil, and prior to sterilization:

1. Immediately following use, ensure that the instruments are wiped down to remove all visible soil and kept from drying by submerging or covering with a wet towel.
2. Disassemble all instruments that can be disassembled.
3. Rinse the instruments under running tap water to remove all visible soil. Flush the lumens a minimum of 3 times, until the lumens flush clean.
4. Prepare Enzo<sup>®</sup> (or a similar enzymatic detergent) per manufacturer's recommendations.
5. Immerse the instruments in the detergent and allow them to soak for a minimum of 2 minutes.
6. Use a soft bristled brush to thoroughly clean the instruments. Use a pipe cleaner for any lumens. Pay close attention to hard to reach areas.
7. Using a sterile syringe, draw up the enzymatic detergent solution. Flush any lumens and hard to reach areas until no soil is seen exiting the area.
8. Remove the instruments from the detergent and rinse them in running warm tap water.
9. Prepare Enzo<sup>®</sup> (or a similar enzymatic detergent) per manufacturer's recommendations in an ultrasonic cleaner.
10. Completely immerse the instruments in the ultrasonic cleaner and ensure detergent is in lumens by flushing the lumens. Sonicate for a minimum of 3 minutes.
11. Remove the instruments from the detergent and rinse them in running deionized water or reverse osmosis water for a minimum of 2 minutes.
12. Dry instruments using a clean soft cloth and filtered pressurized air.
13. Visually inspect each instrument for visible soil. If visible soil is present, then repeat cleaning process starting with Step 3.

## CONTACT INFORMATION

Globus Medical may be contacted at 1-866-GLOBUS1 (456-2871). A surgical technique manual may be obtained by contacting Globus Medical.

## STERILIZATION

These implants may be available sterile or nonsterile. HA-coated implants are only available sterile. Instruments are available nonsterile.

Sterile implants are sterilized by gamma radiation, validated to ensure a Sterility Assurance Level (SAL) of  $10^{-6}$ . Sterile products are packaged in a heat-sealed double pouch or container/pouch. The expiration date is provided in the package label. These products are considered sterile unless the packaging has been opened or damaged. Sterile implants meet pyrogen limit specifications.

Nonsterile implants and instruments have been validated to ensure an SAL of  $10^{-6}$ . The use of an FDA-cleared wrap is recommended, per the Association for the Advancement of Medical Instrumentation (AAMI) ST79, *Comprehensive Guide to Steam Sterilization and Sterility Assurance in Health Care Facilities*. It is the end user's responsibility to use only sterilizers and accessories (such as sterilization wraps, sterilization pouches, chemical indicators, biological indicators, and sterilization cassettes) that have been cleared by the FDA for the selected sterilization cycle specifications (time and temperature).

When using a rigid sterilization container, the following must be taken into consideration for proper sterilization of Globus devices and loaded graphic cases:

- Recommended sterilization parameters are listed in the table below.
- Only FDA-cleared rigid sterilization containers for use with pre-vacuum steam sterilization may be used.
- When selecting a rigid sterilization container, it must have a minimum filter area of 176 in<sup>2</sup> total, or a minimum of four (4) 7.5in diameter filters.
- No more than one (1) loaded graphic case or its contents can be placed directly into a rigid sterilization container.
- Stand-alone modules/racks or single devices must be placed, without stacking, in a container basket to ensure optimal ventilation.
- The rigid sterilization container manufacturer's instructions for use are to be followed; if questions arise, contact the manufacturer of the specific container for guidance.
- Refer to AAMI ST79 for additional information concerning the use of rigid sterilization containers.

For implants and instruments provided NONSTERILE, sterilization is recommended (wrapped) as follows:

| Method | Cycle Type | Temperature   | Exposure Time | Drying Time |
|--------|------------|---------------|---------------|-------------|
| Steam  | Pre-vacuum | 132°C (270°F) | 4 minutes     | 45 minutes  |

*These parameters are validated to sterilize only this device. If other products are added to the sterilizer, the recommended parameters are not valid and new cycle parameters must be established by the user. The sterilizer must be properly installed, maintained, and calibrated. Ongoing testing must be performed to confirm inactivation of all forms of viable microorganisms.*

**CAUTION:** Federal (USA) Law Restricts this Device to Sale by or on the order of a Physician.